# DEPPELER®

S W I S S D E N T A L  
M A N U F A C T U R E 

Précision | Tradition | Innovation

## Table des matières

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## Indice

Table of content

---

Introduction	Einführung	Introduzione	Introduction	01
Les manches	Die Griffe	I manici	The handles	02
Diagnostic	Diagnostik	Diagnostica	Diagnostic	03
Parodontologie	Parodontologie	Parodontologia	Periodontology	09
Aiguillage	Schleifen	Affilatura	Sharpening	23
Chirurgie	Chirurgie	Chirurgia	Surgery	27
Endodontie	Endodontie	Endodonzia	Endodontic	37
Traitement conservateur	Zahnerhaltung	Conservativa	Restorative	43
Cassettes de stérilisation	Sterilisationskassetten	Cassette di sterilizzazione	Sterilization boxes	57
Entretien	Wartung	Mantenimento	Maintenance	62
Montage d'ADEP	ADEP Aufbereitung	Montaggio di ADEP	ADEP mounting	63
Index par article	Index pro Artikel	Indice per articolo	Index per article	64
Index alphabétique	Alphabetischer Index	Indice alfabetico	Alphabetical index	66

## Introduction

---

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---

**Serious instruments for serious work!**

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---

The only manufacturer of its kind, Deppeler SA has been manufacturing dental instruments with unprecedented accuracy for over 80 years.

Basing ourselves daily on our traditional values, precision and innovation, we are able to offer you instruments tailored to their function and to meet your expectations.

The use of exceptional raw materials and the rigor of our work are the key parameters for the success of our mission.

Inventions include the M23<sup>®</sup> scaler, the colour coding of Gracey curettes, treatment kits for titanium implants; our job is to imagine, develop and produce first-rate instruments.

"Synergy : a phenomenon whereby several players acting together create more than the sum of the expected effects if they operated independently." This definition perfectly describes our collaboration with dentists, hygienists and professors. We have created many products to facilitate the daily practice of the practitioner.

We always keep in mind that an ideal instrument is one that fits naturally, with which the work is done perfectly and effortlessly : the extension of the practitioner's hand.

We are proud to present herewith our full range

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Fixed colour following the Deppeler colour code.  
 Available only for instruments illustrated in this catalogue with a CN handle.

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Free choice of colour.  
 Available for most instruments. These handles are not removable.

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Instruments marked with a\* are only available in the classic version.

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Mirrors and screw tips handle, made of plastic. Available in green or black. Ergonomic and comfortable.

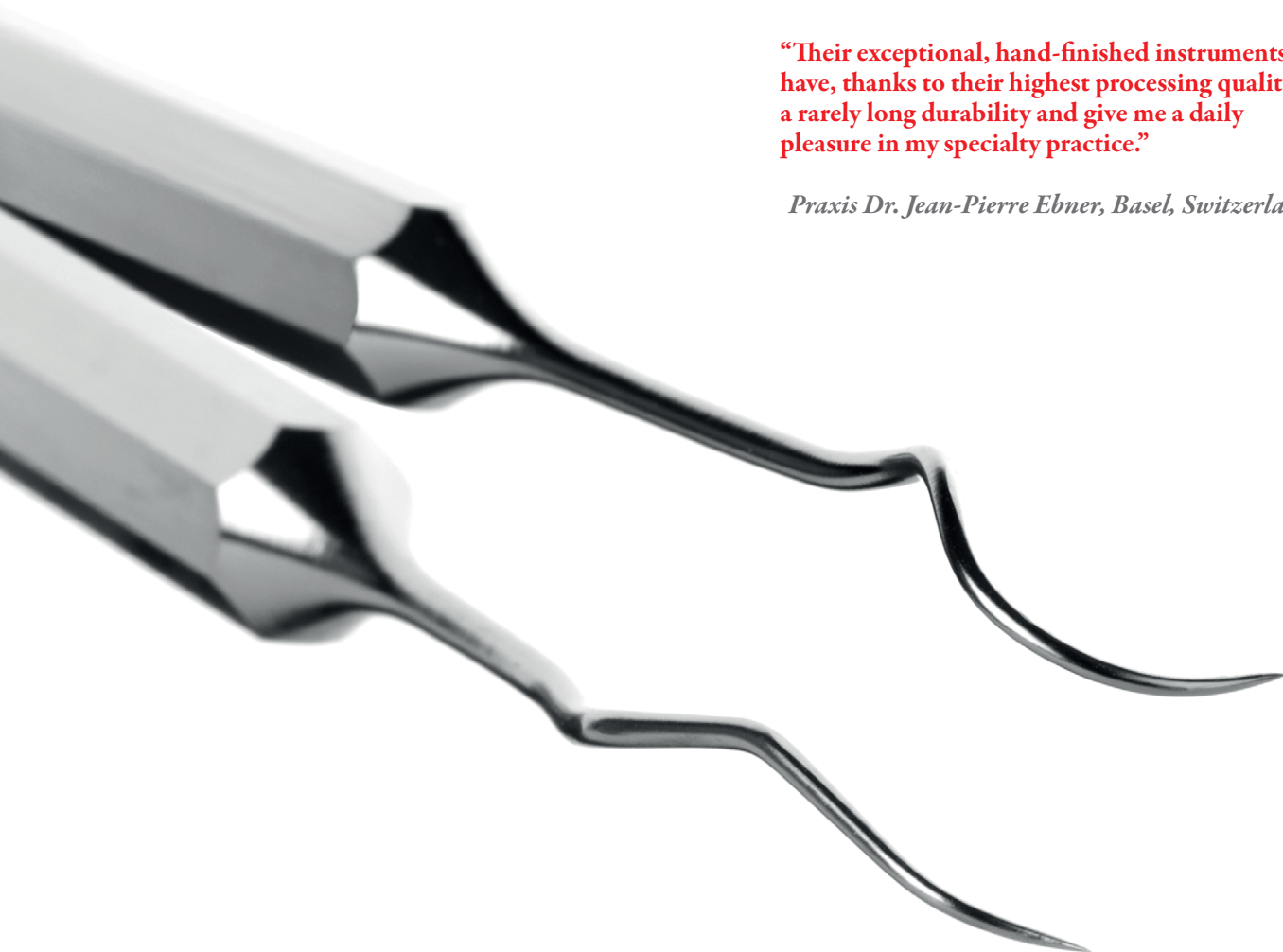
Diagnostic  
 Diagnostik  
 Diagnostica  
 Diagnostic

---

Miroirs	Spiegel	Specchietti	Mirrors	04
Sondes	Sonden	Sonde	Explorers	04
Sondes parodontales	Paro Sonden	Sonde parodontali	Periodontal probes	05
> Sonde à implants	> Implantat Sonde	> Sonda per impianti	> Implant probe	05
> Sondes doubles	> Doppelendige Sonden	> Sonde doppie	> Double-end probes	05
> Sondes gravées	> Gravierte Sonden	> Sonde incisive	> Engraved probes	05
> Sonde Biotyp	> Biotyp Sonde	> Sonda Biotipo	> Biotype probe	06
> Sondes simples	> Einendige Sonden	> Sonde singole	> Single-end probes	06
Sondes à furcations	Furkations Sonden	Sonde per forcazioni	Furcations probes	07
Précelles	Pinzetten	Pinzette	Tweezers	07

**“Their exceptional, hand-finished instruments have, thanks to their highest processing quality, a rarely long durability and give me a daily pleasure in my specialty practice.”**

*Praxis Dr. Jean-Pierre Ebner, Basel, Switzerland*

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
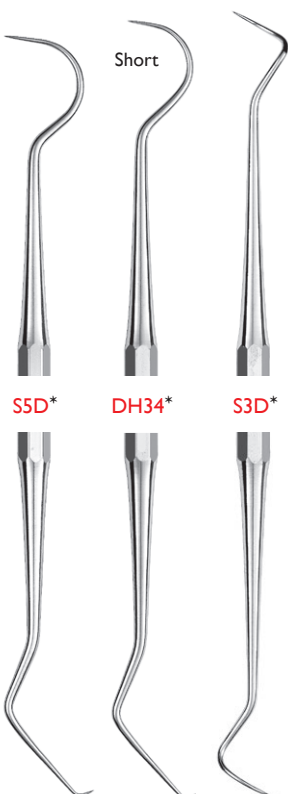
Thanks to the unique characteristics of our raw material, our explorers have an unmatched lifetime. Each working end is hand-finished to guarantee a perfectly sharp tip.

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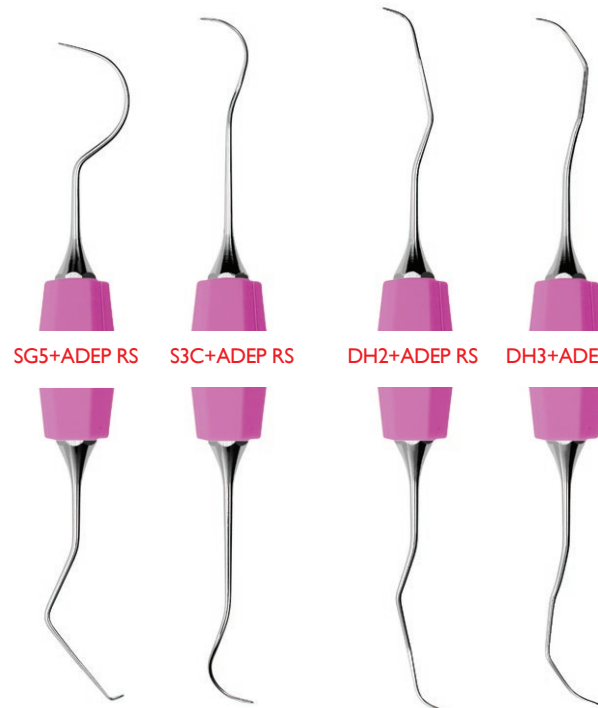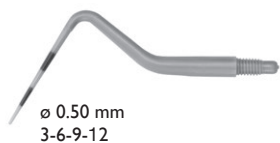
- Adapts to the anatomy due to its flexibility
- indelible precise marking through the material
- also suitable for exploration of natural teeth.

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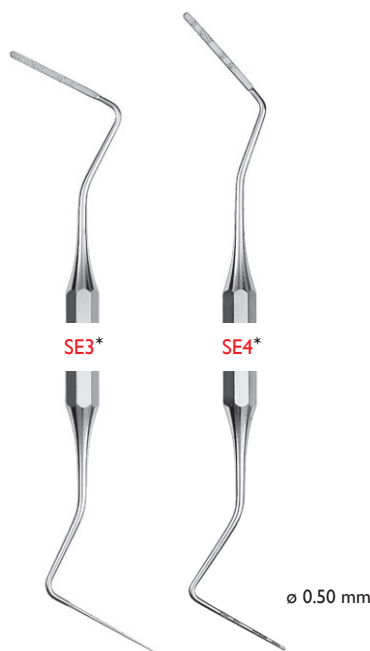
Biotype probe

Clinical use :

1. If the fine / graduated part is visible through the gingiva, then it is a very thin biotype.
2. If the fine / graduated part is not visible but the thick / black part is, then it is a medium biotype.
3. If the thick / black part is not visible through the gingiva, then it is a very thick biotype.

Fischer KR et al. Gingival biotype revisited – novel classification and assessment tool.

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Single-end probes

Our Perio Probes combine flexibility and resilience. Pocket depth markings will last permanently.

> Pointes à visser parfaitement arrondies  
 Perfekt abgerundete Spitzen zum einschrauben  
 Punte a vite perfettamente arrotondate  
 Screw tips with perfectly rounded ends

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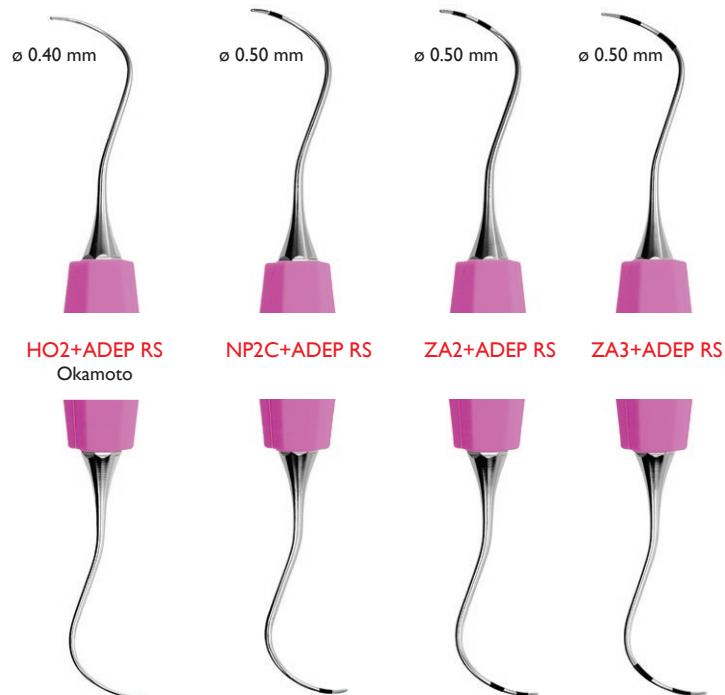
The furcation probes are not a measuring device.

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Universal curettes & scalers M23<sup>®</sup>, M23F, M23A<sup>®</sup>, M23AS

## The most universal Scaler

**"The M23<sup>®</sup> is a unique double-ended scaler, which can be easily adapted for treatment in posterior and anterior by simply changing the operator sitting position."**

*Bernita Bush Gissler, RDH,BS,  
Bern, Switzerland*

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### Different versions

Try our fine versions for greater sensitivity in your movements and better adaptation to narrow anatomies. Not only the blade, but the entire shaft is thinner.

	Standard	Fine
Scaler	M23 <sup>®</sup>	M23F
Curette	M23A <sup>®</sup>	M23AS

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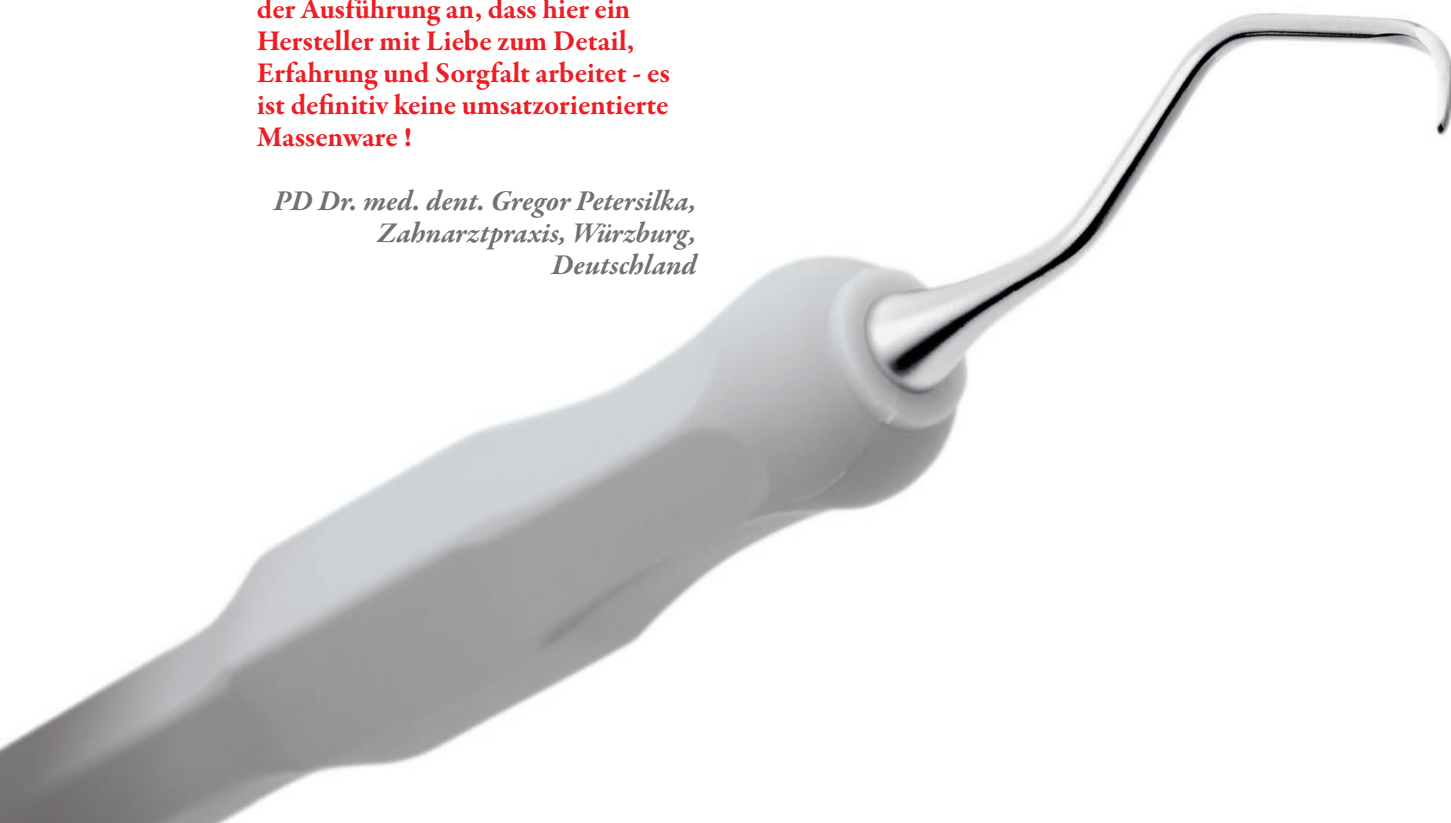
- All root surfaces can be treated with two instruments only
- Revolutionary angulation and blade allow for perfect access to periodontal pockets
- Depth graduation improves precision and safety

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“Modern hand instruments should meet the requirements of minimal invasive treatment. Therefore they need to be precise and delicate, whilst retaining strength. Besides ergonomic considerations, they should enable the most efficient clinical work. By working together with Deppeler on the development of the new DSS curettes, we have fulfilled these standards. The double sided cutting edges and the special angulation of the tip make it possible to access all surfaces of the dentition with only two instruments. The periodontal “probe like” pocket depth markers on the end of the instruments enable a local precise debridement without damaging healthy tissue. The newly designed CN handle reduces operative fatigue. Try the DSS curettes, you will be enthusiastic!”

Daniela Krejci, Dental hygienist HF, CDS Signy, Switzerland

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
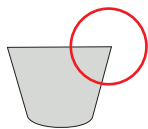
The Deppeler Smart Scaling blade provides a double edge with a wider angle and a rounded conical shape. This very special geometry allows to perform work safely without damaging the healthy tissue.
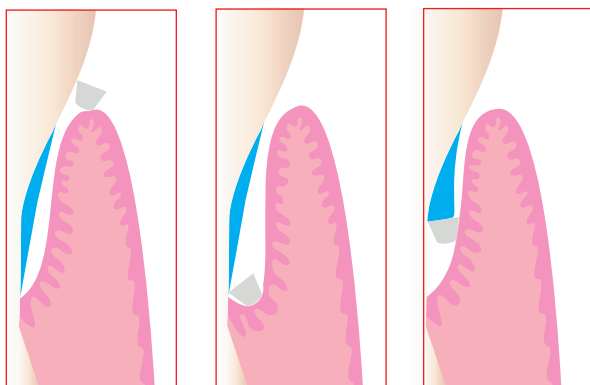
> **Angle plus ouvert = sécurité de traitement**  
Offenerer Winkel = Sicherheit bei der Behandlung  
**Angolo più aperto = sicurezza di trattamento**  
Wider angle = safe treatment

> **Lame conique pour un accès optimal.**  
Konische Klinge für einen optimalen Zugang.  
**Lama conica per un accesso ottimale.**  
Conical blade for an optimal access.

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Laser marking on the shaft ensures that the clinician is working at the proper depth. For the first time, the practitioner works safely and with unprecedented precision. Do not lose your measurement! Stay precise!

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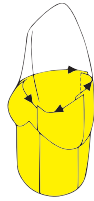
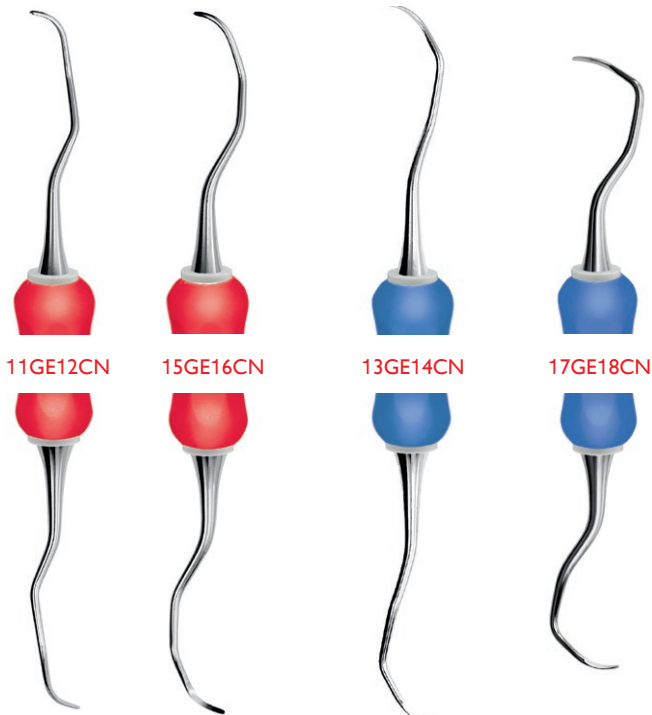
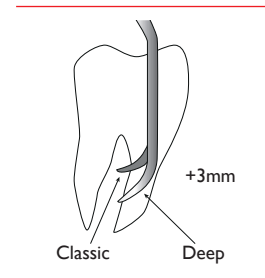
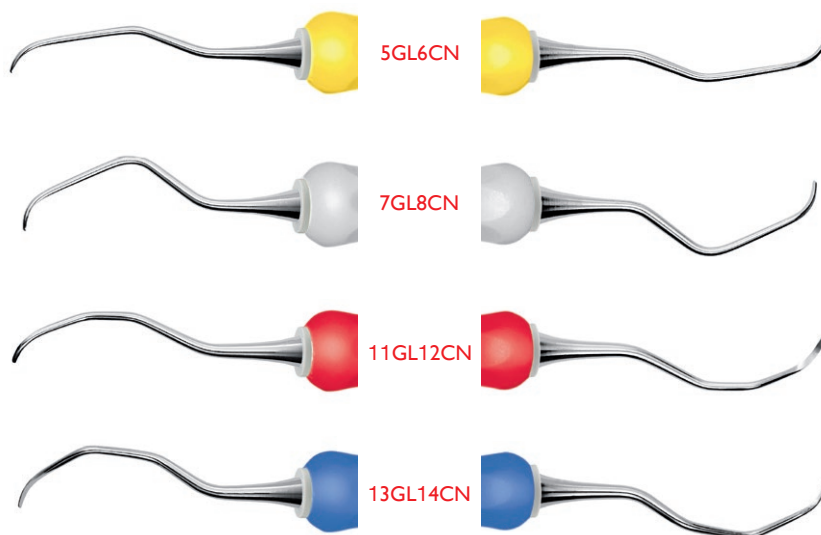
## Deep

This range of Gracey offers more comfort to both the patient and the practitioner.

The best solution for deep pockets, thanks to :

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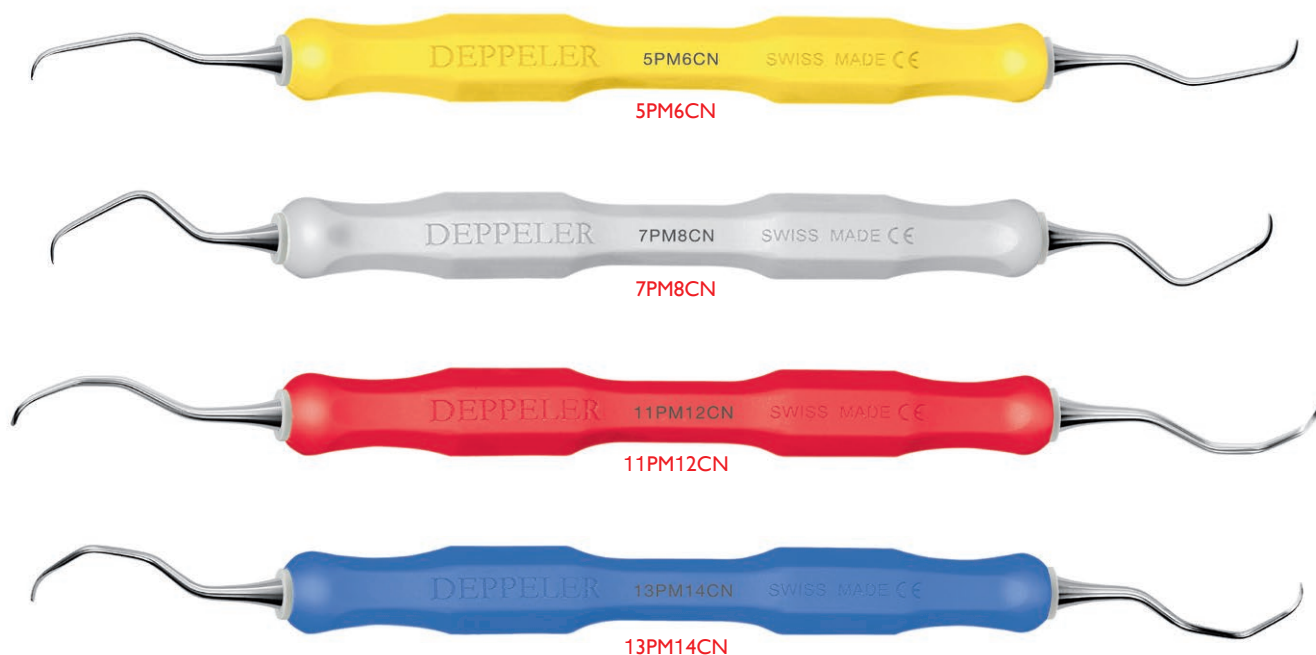
## Mini Profil

Shorter and finer blades, optimal shaft and blade curvature enable an almost non-invasive curetting as the curettes closely match root profile and irregularities.

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## Nano Profil

The short and fine blades of these curettes allow a very slight invasive curettage. Furthermore, they fit the finest irregularities of the root and their perfect cutting edge offers a high efficiency.

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## Rigid

Larger shaft diameter. Blade identical to the Classic.

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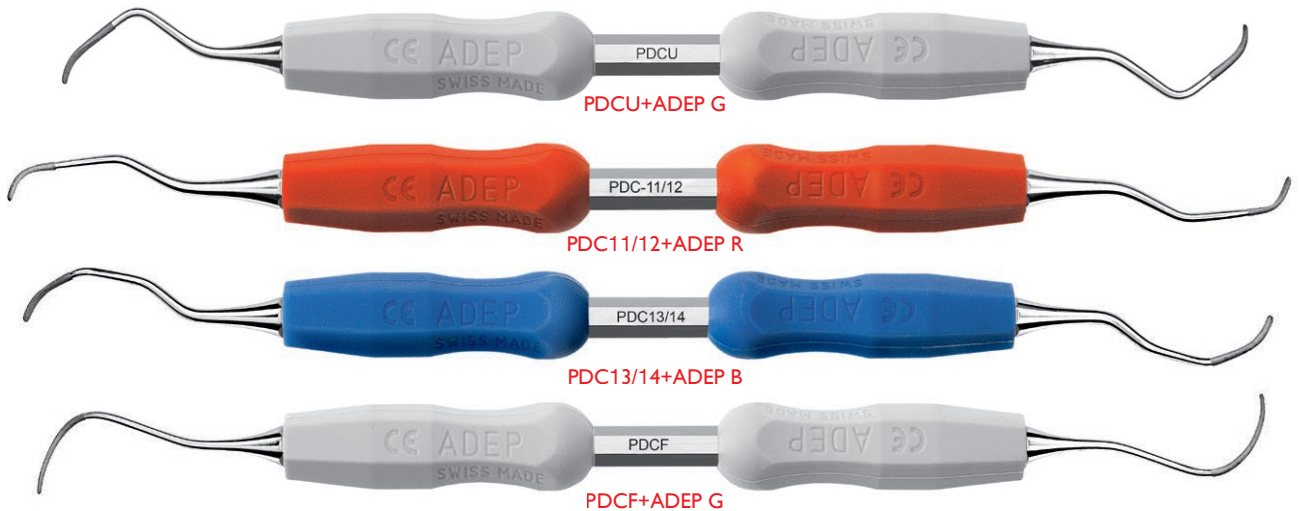
### Diamond coated

"Treatment of root surfaces with conventional Gracey curettes followed by subsequent polishing with diamond-coated curettes may result in a root surface which favors the attachment of PDL (periodontal ligaments) fibroblasts but not the microbial adhesion."\*

### >

PerioDiaCurette

Dr. Paolo Siervo, Milano, Italia

> \*Eick, Bender, Flury, Lussi, Sculean (2012) In vitro evaluation of surface roughness, adhesion of periodontal ligament fibroblasts, and Streptococcus gordonii following root instrumentation with Gracey curettes and subsequent polishing with diamond-coated curettes. Clin Oral Invest 14:4

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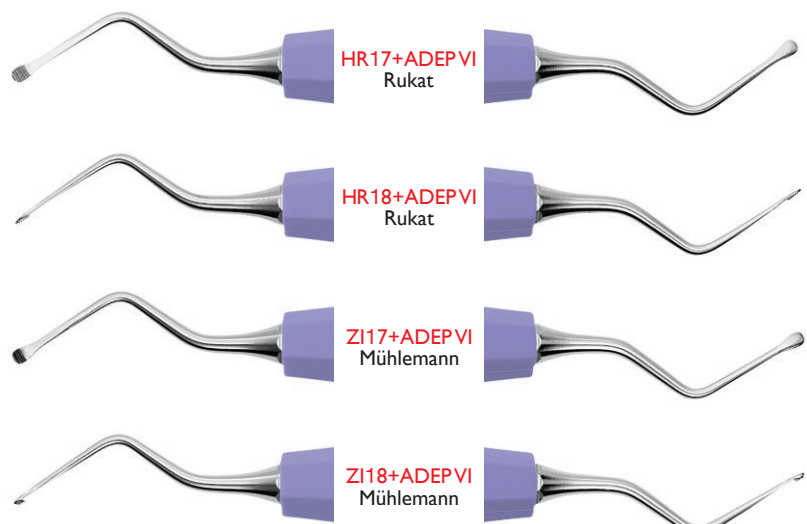
## Root finishing files



Very thin files, with fine teeth for easy penetration and an optimal smoothing of the root.

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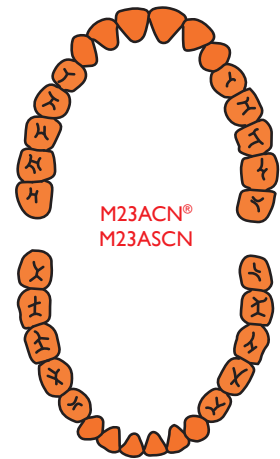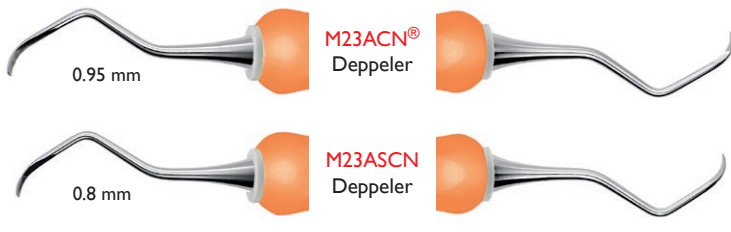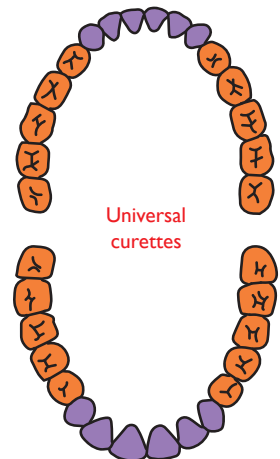
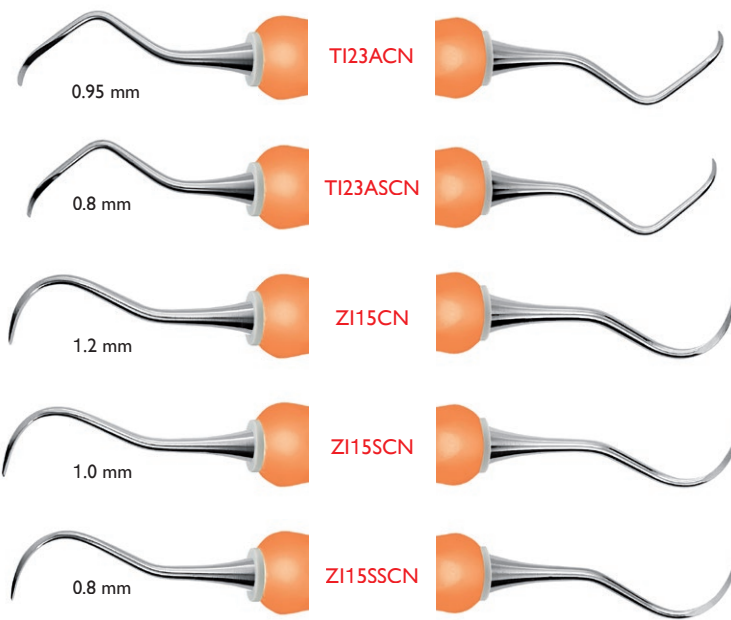
Fine versions

These fine curettes enable extremely delicate debridement.

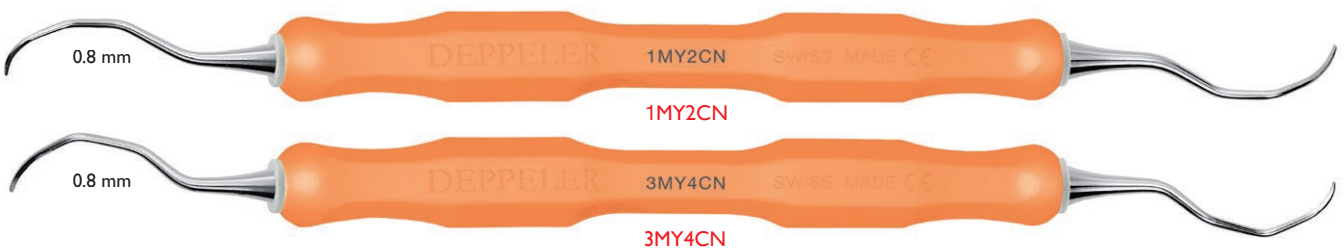
**IMPORTANT:** not only the blade, but the entire shaft is thinner. This allows greater sensitivity during the clinical work. It is also a little less strong.

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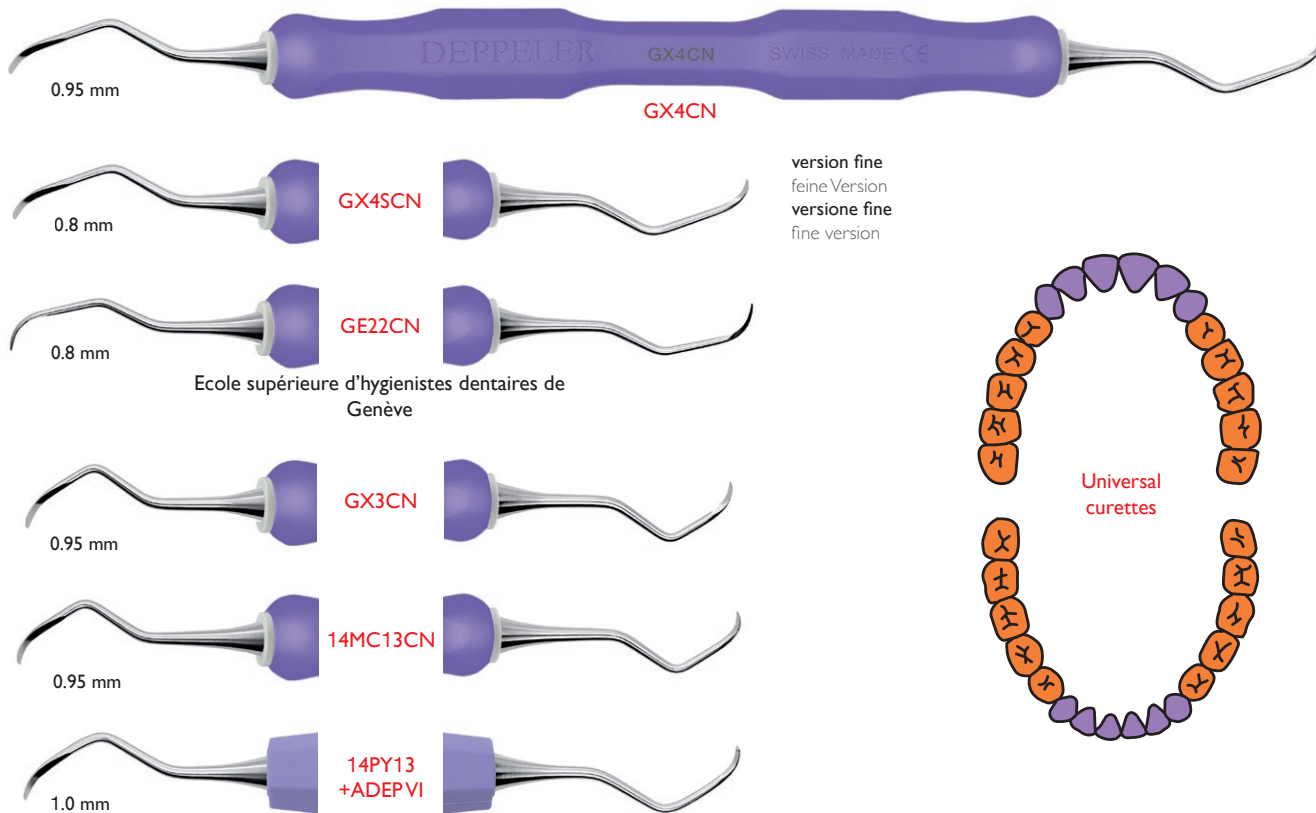
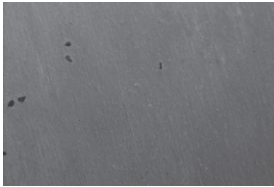
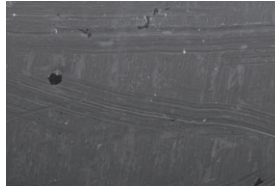
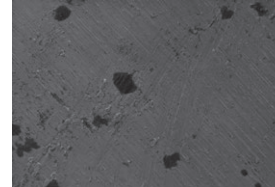
Efficient without damaging

These solid titanium curettes are softer than the titanium used for the production of implants.

Surface of a titanium implant under the microscope.\*

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2 instruments for all clinical cases.

The new designed Titanium instruments from Deppeler are a class of their own. Better access to tight soft tissue niches or areas around the superstructure is achieved due to the development of a smaller yet durable working end.

Bernita Bush Gissler, RDH,BS, Bern, Switzerland

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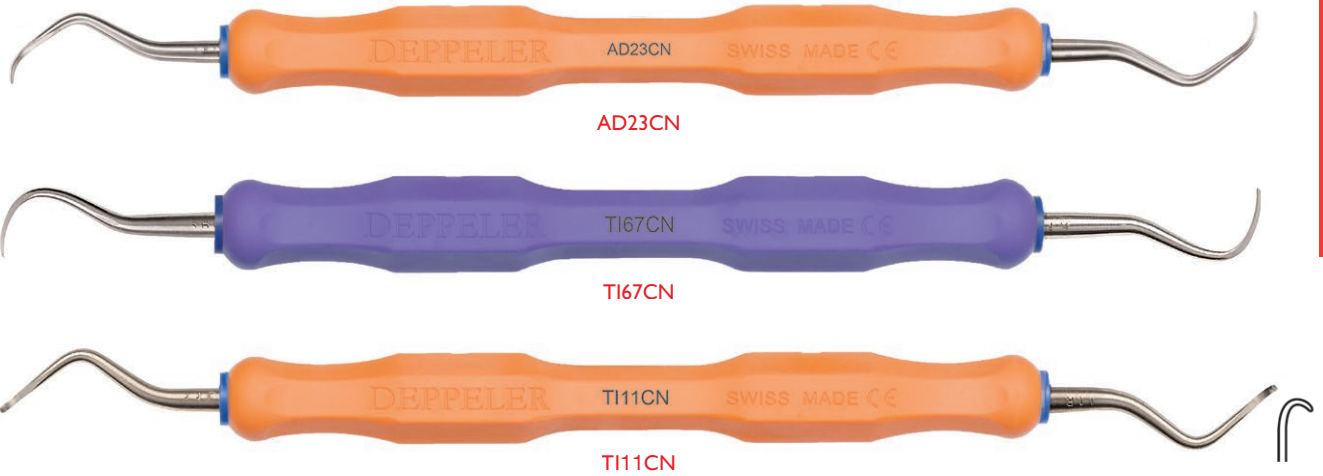
Tip shape equivalent to M23A®, allows to treat all surfaces.

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Tip shape equivalent to H6H7, with a rounded tip.

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Shape adapted to the treatment of approximal areas.

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The periodontal probes are not a measuring device.

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Sharpening stone for titanium curettes

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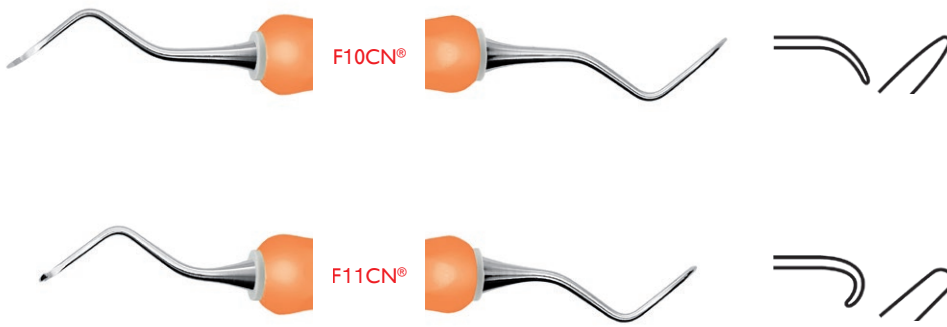
2 steps for clean furcations

1. Curettage with the curettes F10 or F11 depending on the size of the furcation (see pictures).

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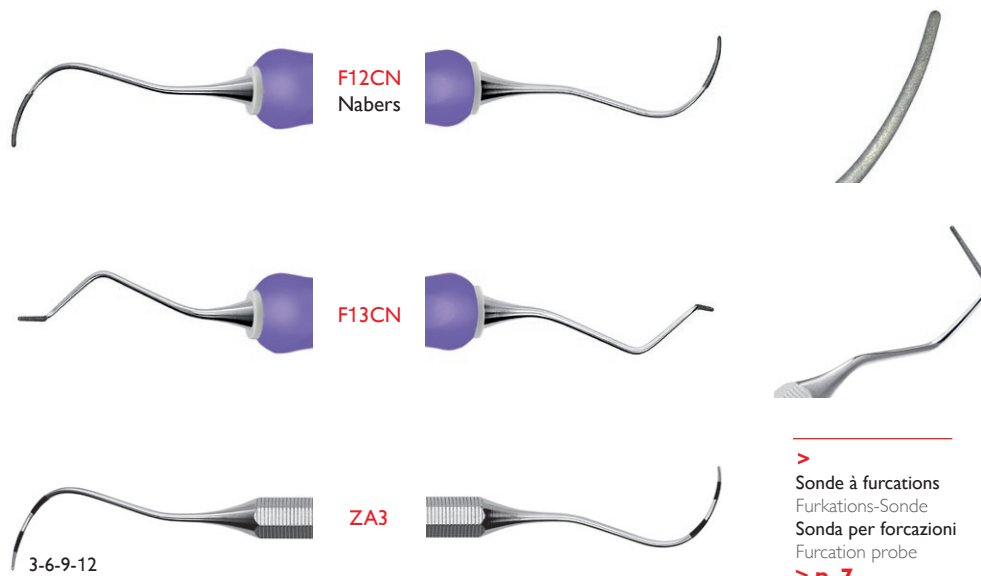
2. Smoothing with diamond coated curettes F12 or F13 depending on the anatomy of the furcation. Very fine diamond grain not to damage the root surface.  
> clinical study p.15

> Sonde à furcations  
Furkations-Sonde  
Sonda per forcazioni  
Furcation probe  
> p. 7

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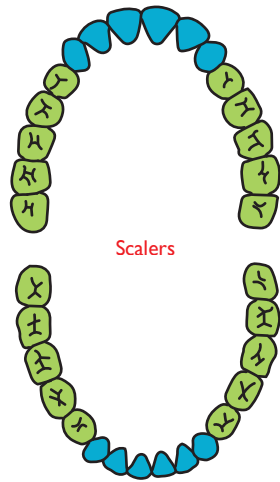
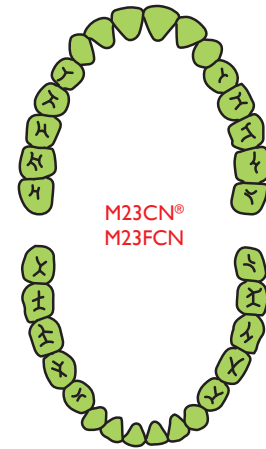
Our mastery of the raw material not only gives to the instruments their incomparable cut, but also an extremely hard-wearing.

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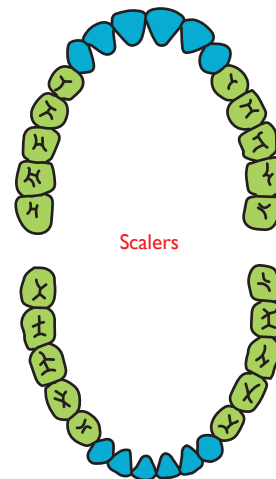
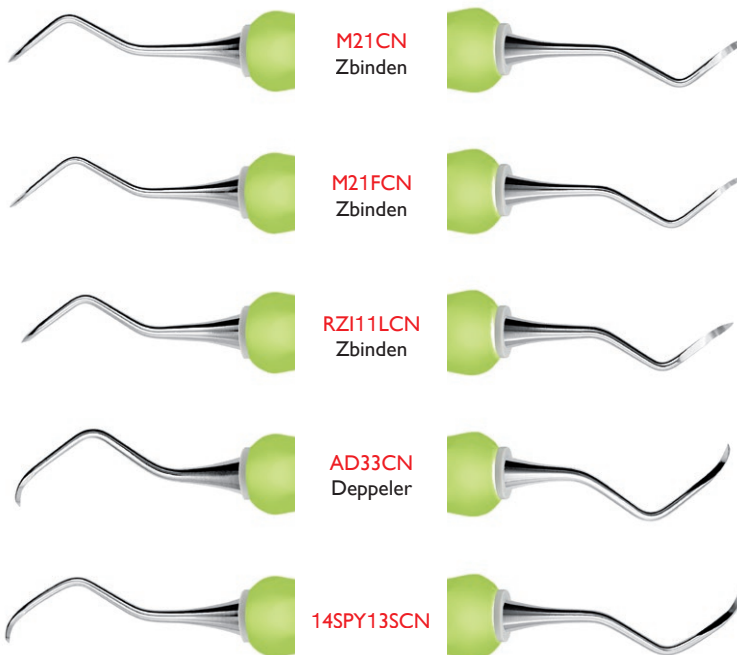
### Fine versions

These fine scalers enable extremely delicate debridement.

**IMPORTANT:** not only the blade, but the entire shaft is thinner. This allows greater sensitivity during the clinical work. It is also a little less strong.

Aiguillage	Schleifen	Affilatura	Sharpening	
> Easy Sharp the device	> Easy Sharp the device	> Easy Sharp the device	> Easy Sharp the device	24
> Easy Sharp the coach	> Easy Sharp the coach	> Easy Sharp the coach	> Easy Sharp the coach	24
> Easy Sharp the stones	> Easy Sharp the stones	> Easy Sharp the stones	> Easy Sharp the stones	25
> Easy Sharp the sticks	> Easy Sharp the sticks	> Easy Sharp the sticks	> Easy Sharp the sticks	25
> Easy Sharp the oil	> Easy Sharp the oil	> Easy Sharp the oil	> Easy Sharp the oil	25

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
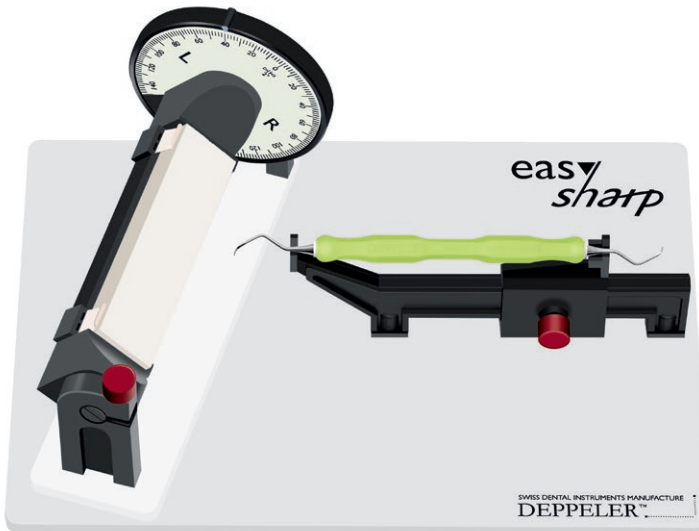
Use your instruments up to 10 times longer :

- maintain a perfect angle
- avoid facets on blades
- keep the same efficiency in all treatments.

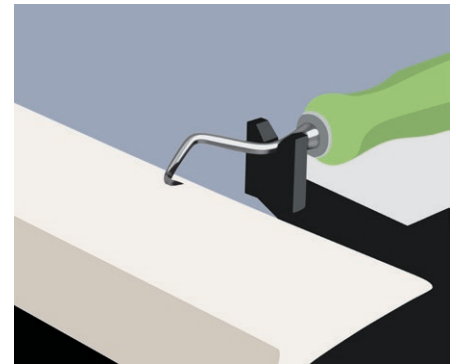**Ref: ESD**  
livré avec 2 pierres  
geliefert mit 2 Steinen  
fornita con 2 pietre  
delivered with 2 stones

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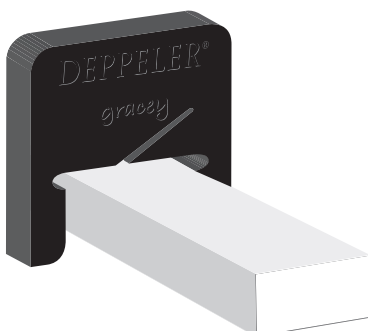
Ideal tool for training in the manual sharpening of instruments :

- perform the correct movement
- sharpen with the right angle
- increase the life of the blades
- avoid facets on blades.

**Ref: ESC**  
contient 2 coaches et 1 pierre ESS1  
enthält 2 Coachs und 1 Stein ESS1  
contiene 2 coach e 1 pietra ESS1  
contains 2 coaches and 1 stone ESS1

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ESS1 to 4 are compatible with Easy Sharp Device & Easy Sharp Coach

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Odourless and colourless oil. Its fluidity allows optimal sharpening.

> **easy**  
*sharp* The sticks

**TEST-Stick**

>

Bâtons test  
paquets de 6 pces  
Teststäbchen  
Packungen von 6 Stk.  
Bastoncini per test  
conf. da 6 pezzi  
Test sticks  
packs of 6 pces

**CLEANext** *technology*

Periotomes	Periotome	Periotomi	Periotomes	28
> Syndesmotomes	> Desmotome	> Sindesmotomi	> Syndesmotomes	28
Précèlles	Pinzetten	Pinzette	Tweezers	28
Raspatoires	Raspatorien	Scollaperiostio	Periosteal elevators	29
> À papilles	> Für Papillen	> Per papille	> For papillae	29
Sinuslift	Sinuslift	Sinuslift	Sinuslift	30
Prichard	Prichard	Prichard	Prichard	30
Limes à os	Knochenfeilen	Lime per osso	Bone files	31
Ciseaux à os	Knochen Meissel	Scalpelli per osso	Bone chisels	31
Râcleur à os	Knochenschaber	Raschietto per osso	Bone scraper	31
Couteaux et lancettes	Gingivectomie Messer	Bisturi parodontali	Periodontal knives	31
Couteaux pour micro-chirurgie	Mikro-Chirurgie Messer	Bisturi per micro-chirurgia	Micro-surgery knives	32
Curettes chirurgicales	Chirurgische Küretten	Curette chirurgiche	Surgical curettes	32
> Curettes fenêtrées	> Öhrlöffel	> Cucchiali cavi	> Eyehole-spoons	32
> Curettes alvéolaires	> Alveolarlöffel	> Cucchiali alveolari	> Spoon-shaped curettes	33
> Curettes planes	> Flache Löffel	> Cucchiali piani	> Flat spoons	33
Back action ciseau	Back action Meissel	Scalpello back action	Back action chisel	33
Tunneling	Tunneling	Tecnica a tunnel	Tunneling	34
Élévateurs	Wurzelheber	Leve	Elevators	36
> Standards	> Standard	> Standard	> Standards	36
> Lames dentelées	> Gezackte Klängen	> Lame dentellate	> Serrated blades	36

**“The beauty of Deppeler instruments is that they are precisely engineered, very robust and of exceptionally high quality - helping clinicians achieve optimal clinical outcomes.”**

*Dr Neesha Patel  
Best Periodontic Practice - Private Dentistry Awards, 2013-2014  
Specialist in Periodontics and Practice Principal - Pure  
Periodontics, London  
Consultant Periodontist - King's College Hospital, London*

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
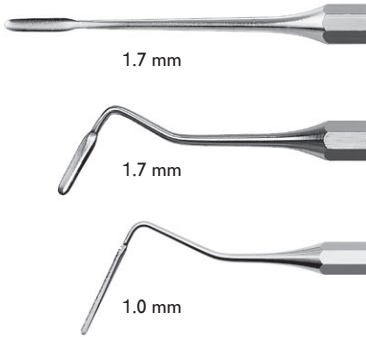
The hand-finished cutting edge enables you to work efficiently and gently. These instruments guarantee you the best possible preparation for an extraction.

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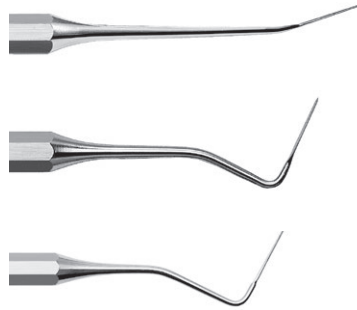These instruments have been designed to cut ligaments before an extraction. Their top quality manufacture and their exceptional cutting edge guarantee precise and efficient work.

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IMPORTANT : these fine instruments should never be used as elevators.

VT3

VT1

VT2

VT4

VT5

VT6

**“Among everything, I love the VT serie: their sharpness is optimal and the edges are extremely well angled. I have finally found the perfect instruments, with the perfect angles.”**

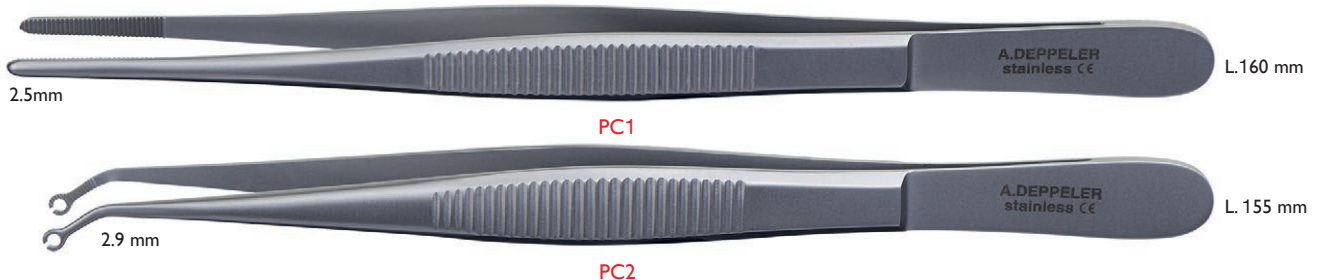
*Dr. med. dent. Masaki Kawase,  
Kawase Dental Clinic, Yatomi-city, Japan*

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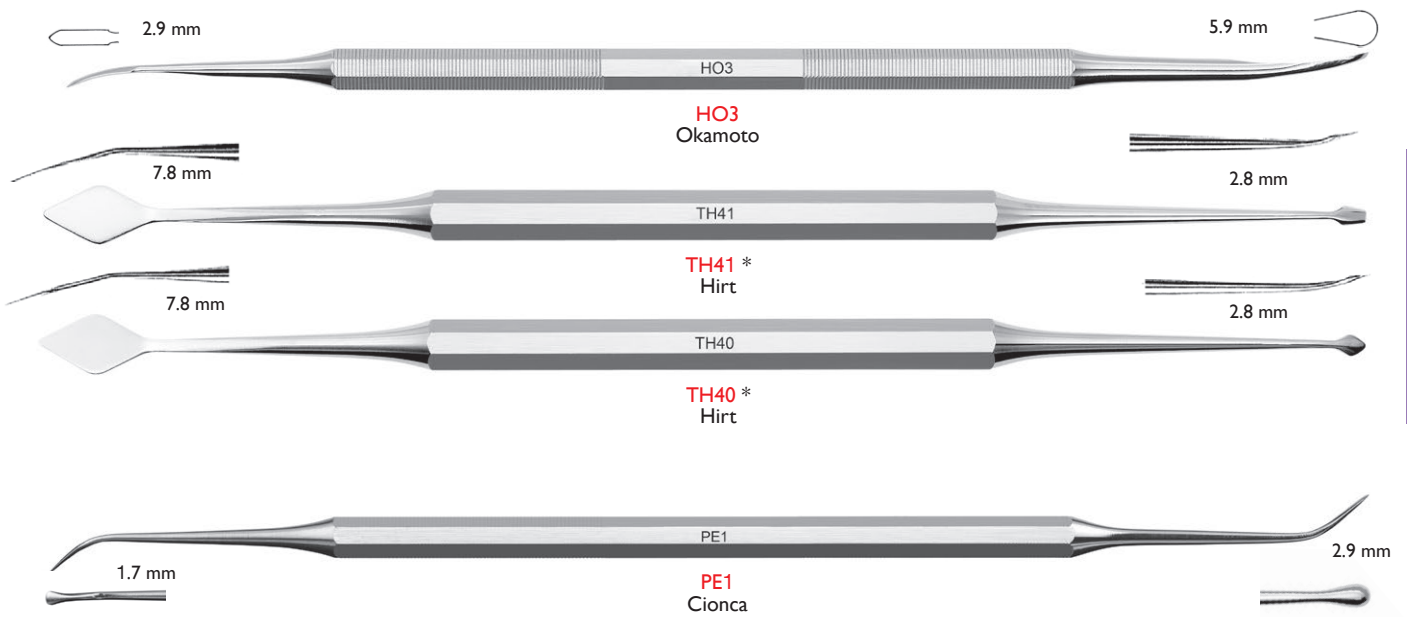
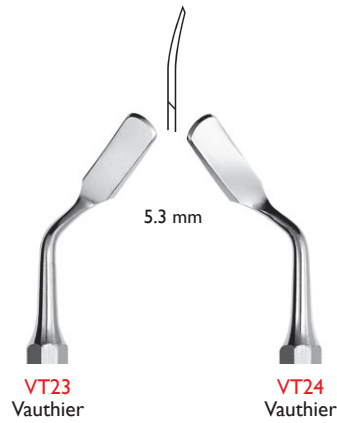
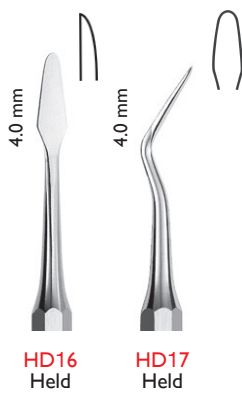
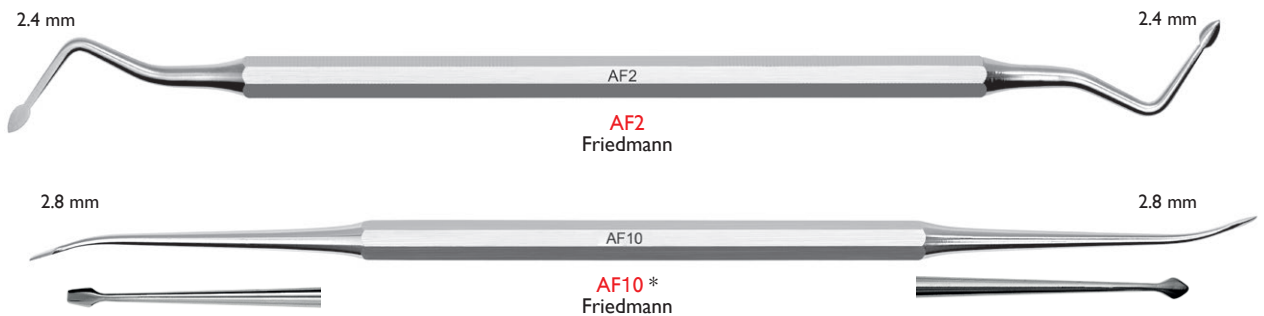
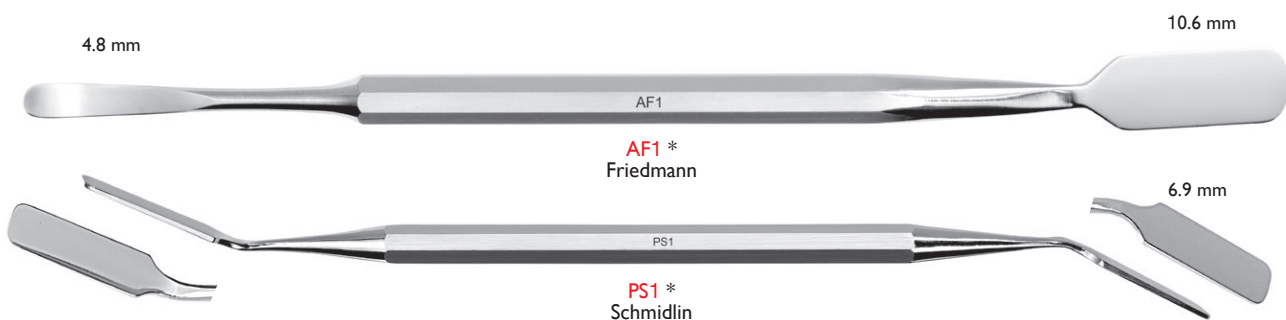
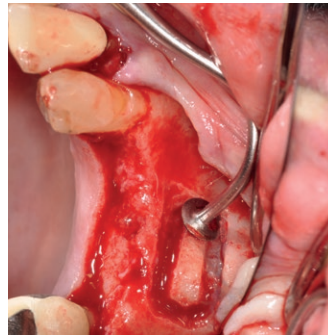
Instruction for use of the SL360 curette

“After having reached the sinus membrane through osteotomy, the release or detachment of the membrane at the edge of the window will be facilitated due to the atraumatic elephant foot curette. Thanks to its geometric form, all sides and edges of the sinus window can be instrumented. In addition, removal of bone spicules will be possible.”

Dr. med. dent. Norbert Cionca,  
Geneva University

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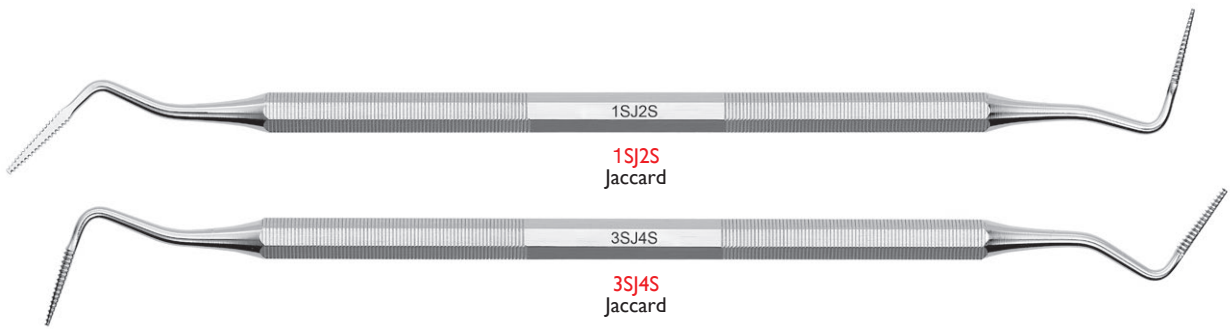
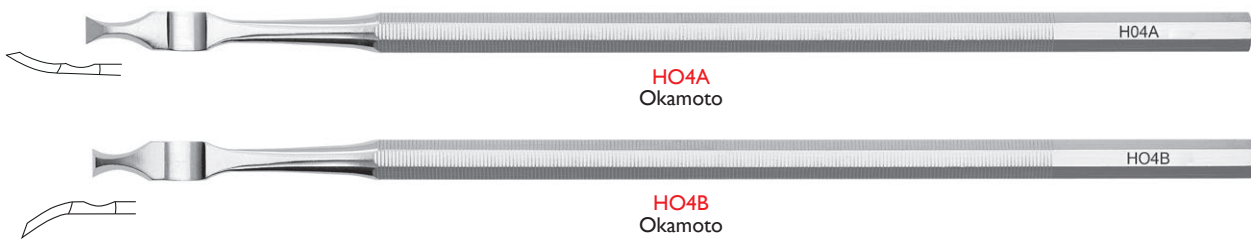
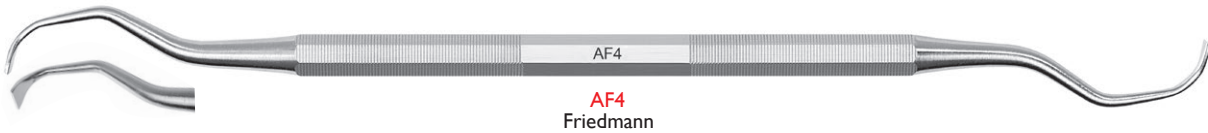
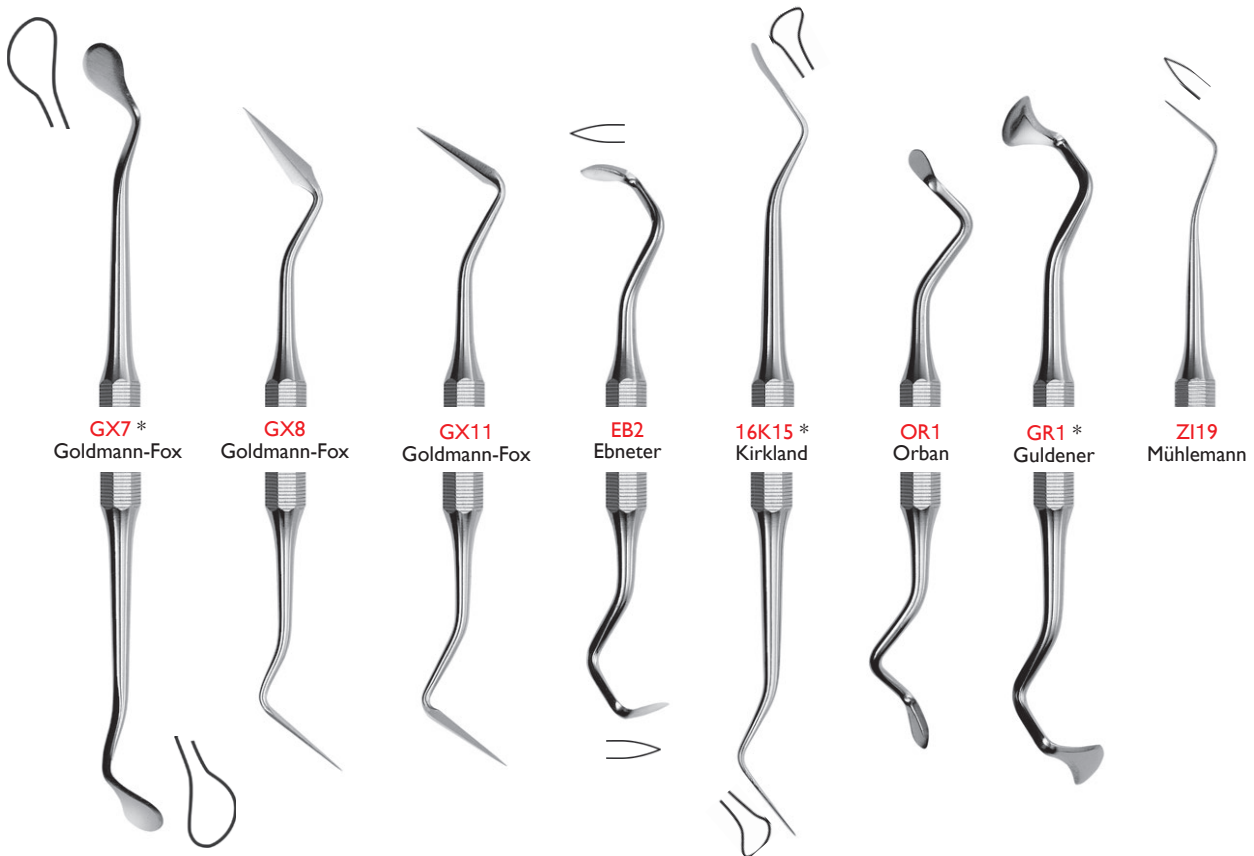
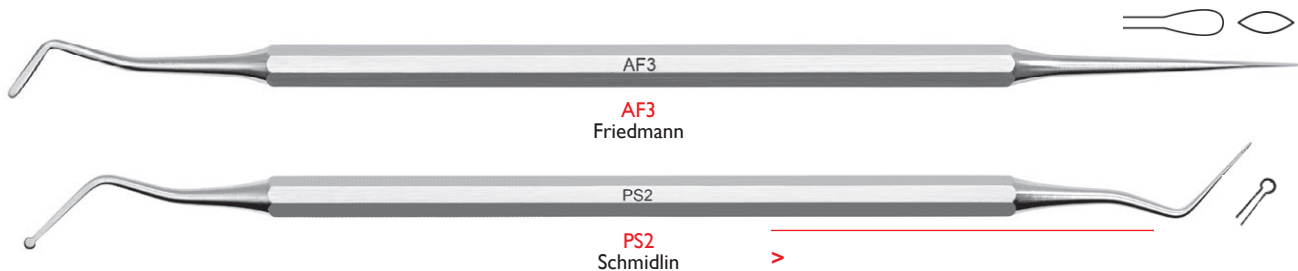
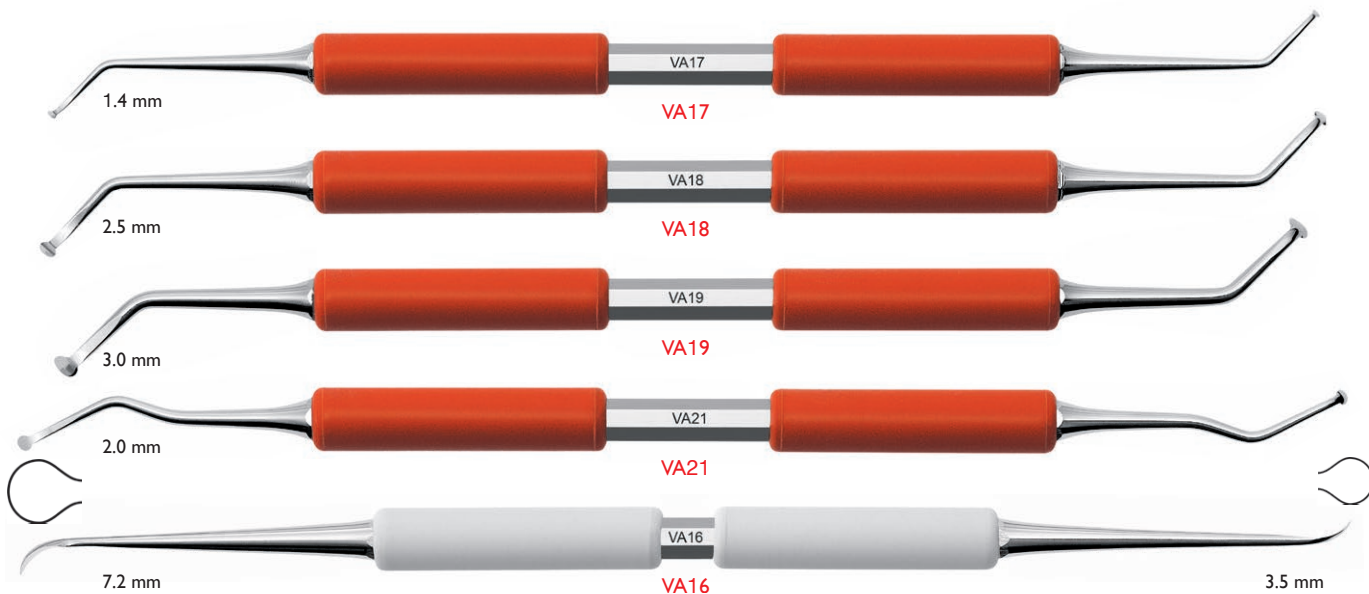
Eye-hole-spoons

These curettes enable an easier ablation of cystes.

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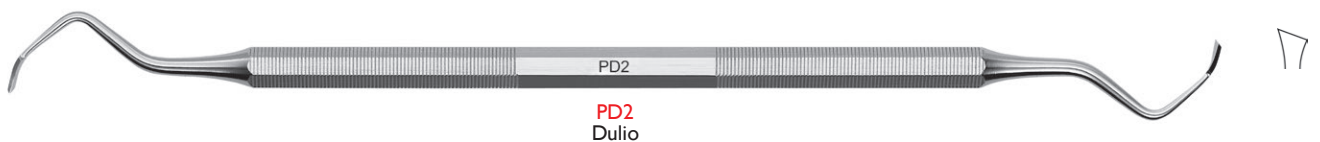
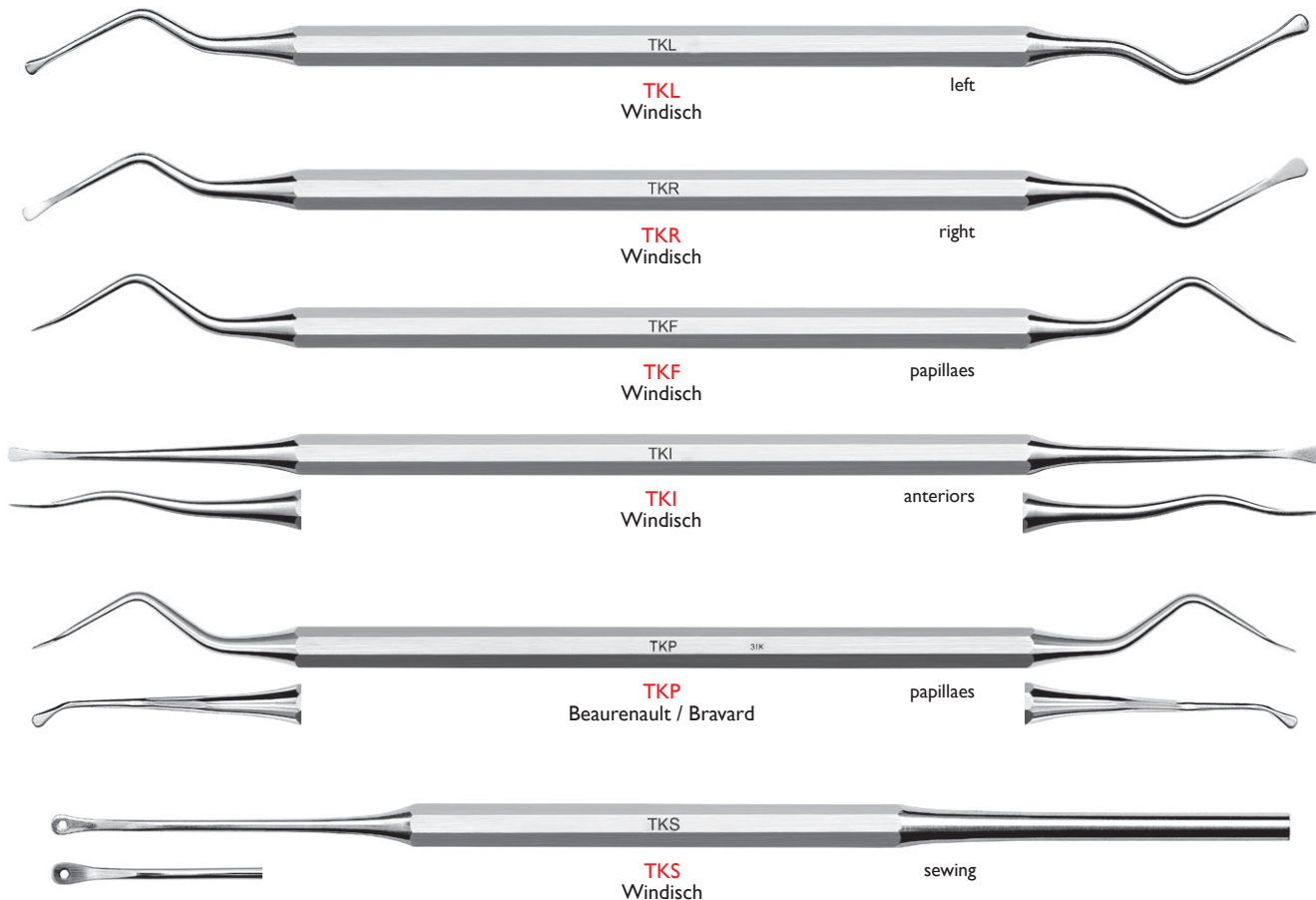
"The Deppeler tunneling knives are exceptionally well designed instruments, perfectly suited for minimalinvasive surgical interventions. The surgical set is ideal for tunnelised flap techniques aiming at root coverage, enlargement of keratinised tissues and socket preservation."

Prof. Péter Windisch /Department of Periodontology, Semmelweis University, Budapest, Hungary

Prof. Dr. med. dent. Péter Windisch

- D.M.D., Semmelweis University Faculty of Dentistry, Budapest, Hungary, 1982-1987
- Associate Professor /Department of Periodontology, Semmelweis University, Budapest, Hungary, 2005-2012
- Professor /Department of Periodontology, Semmelweis University, Budapest, Hungary, 2012-present
- Chairman, Hungarian Society of Periodontology
- Vice chairman, Hungarian Society for Dental Implantology

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Dr. med. dent. Olivier Beurenault

"Dental instrument developed for dissection in profound spaces or in mucoperiosteal flaps without relief incisions. Advantages of the working end design:

- perforation of the flap can be avoided thanks to the 'egg shaped' cutting edge.
- damage to the marginal gingiva is minimized due to the rounded, soft shaft. This would be difficult to achieve with mini-blades.

Technics such as flap exposure and tension free mucoperiosteal flap surgery will be made easier without risk of traumatizing tissues especially when performing microsurgery and tunneling technics."

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This photo shows how a blade shoved along the tooth cervic risking damage and trauma. This danger does not exist when using the LJM.

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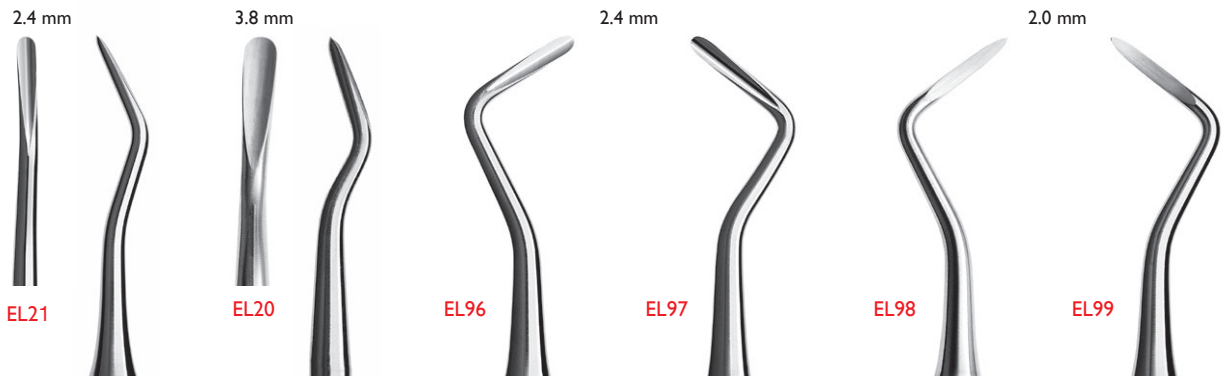
### Standards

Our elevators are strong, lightweight and well suited to their use.

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### Serrated blades

Elevators with serrated blades have more grip.

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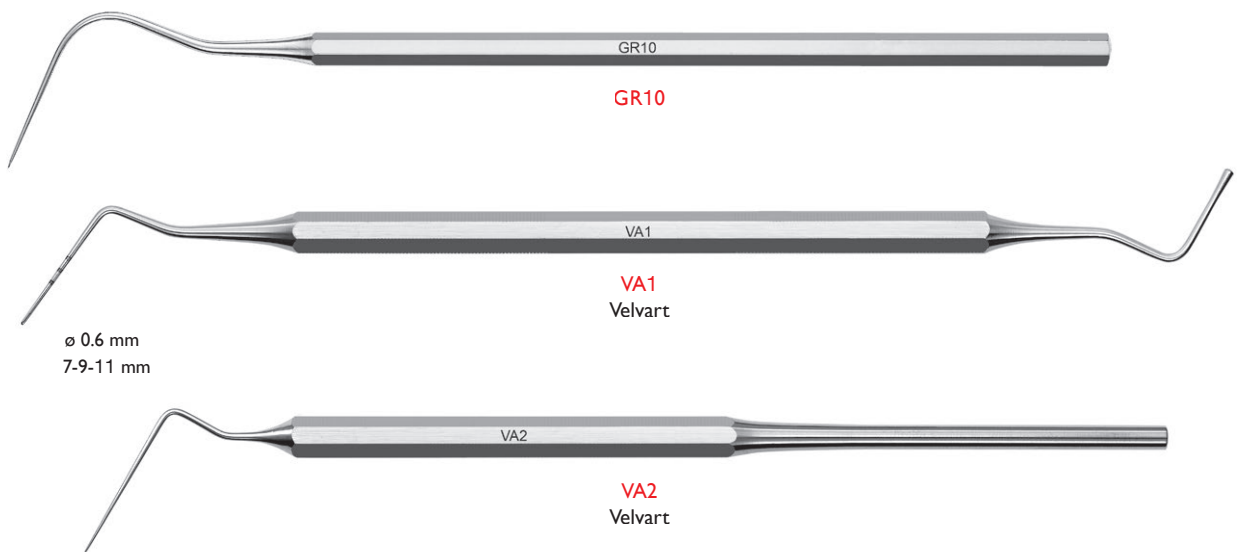
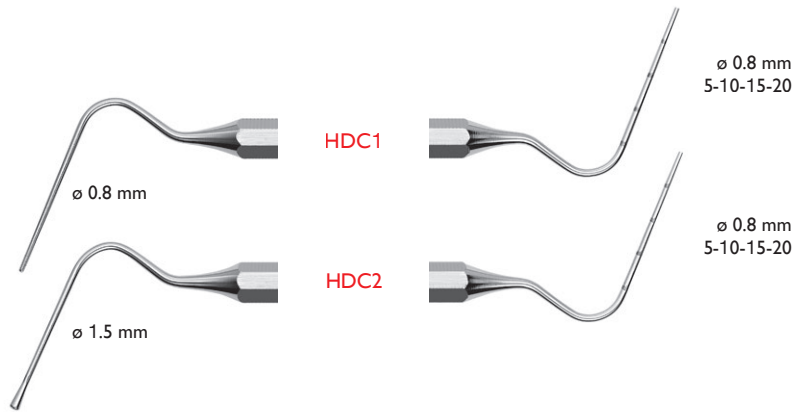
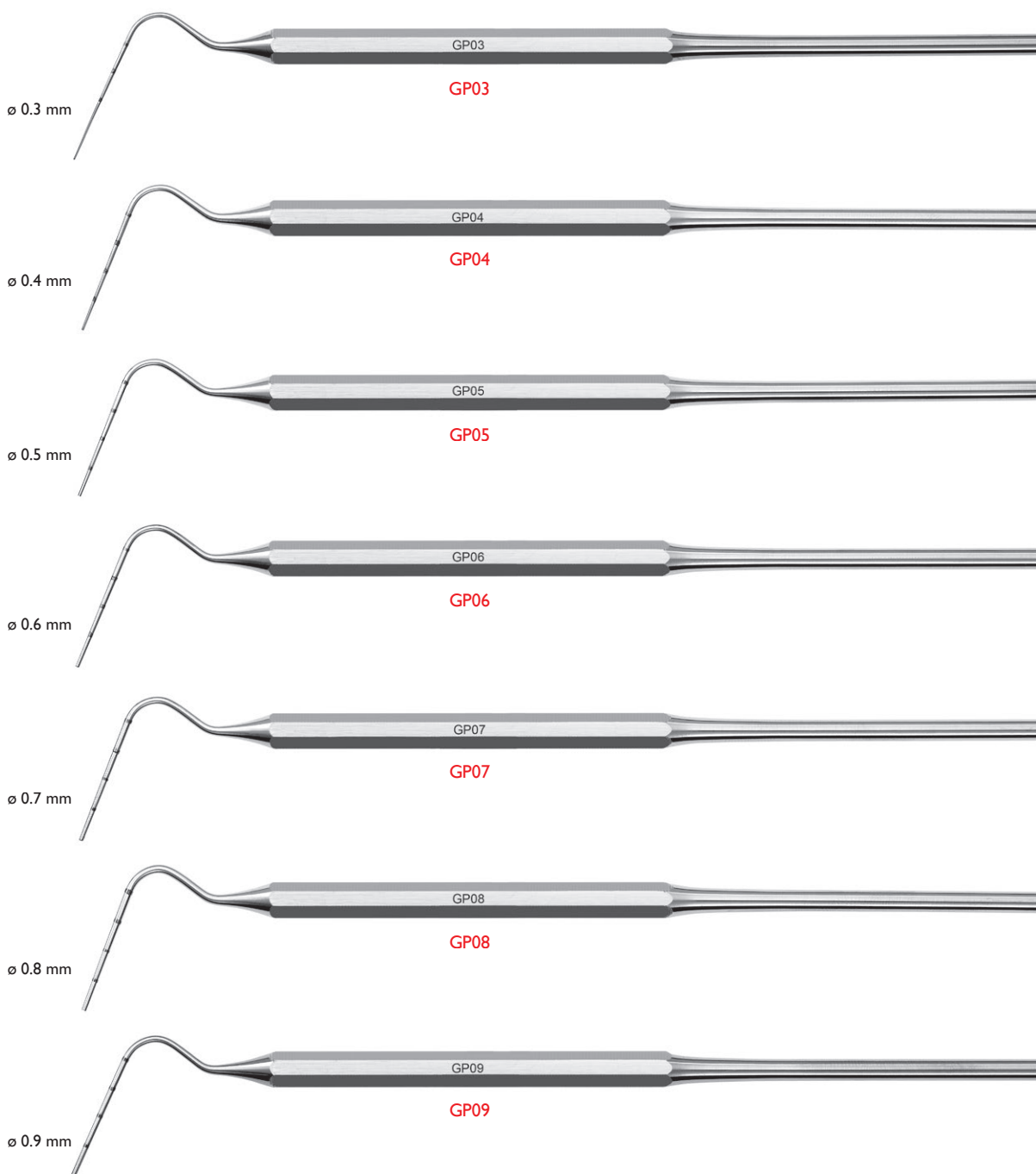
Below, a complete set of pluggers for vertical compaction of filling materials in the radicular canal. Each instrument is graduated 5-10-15-20. The diameter indicated corresponds to the very end of the tip.

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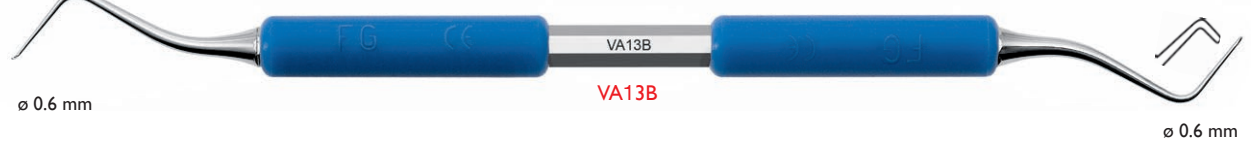
Counter-bent plugger for side teeth

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The Shadeline® models offer an improved coefficient of surface friction and abrasion resistance. The practitioner will profit from the following advantages:

1. improved ease of handling and shaping of diverse composite materials
2. the black color of the patented Shadeline® instruments prevents bothersome reflections seen with steel instruments
3. a longer instrument life than traditional composite instrument.

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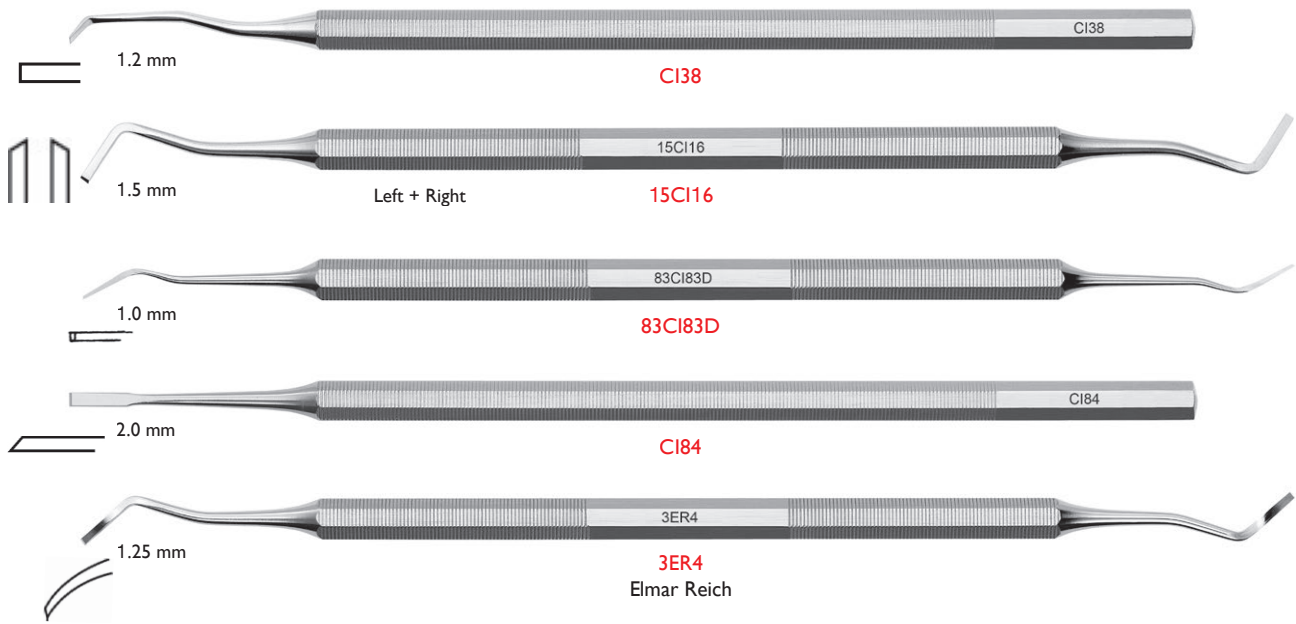
Nazariy Mykhaylyuk, DDS, Dental Clinic of Mykhaylyuk, Ukraine

“These enamel chisels are incredible hand instruments. They give an ability to create perfectly smooth preparation margin (for full crowns, veneers, onlays) for direct and indirect restorations. And as a result - to gain a maximum precision. Three sizes (1.0 1.4 1.6) respond to the same size of burs in Nazariy Mykhaylyuk's preparation kit. So we get a full system of preparation, starting with burs and finishing with chisels.”

Nazariy Mykhaylyuk, DDS, Dental Clinic of Mykhaylyuk, Ukraine

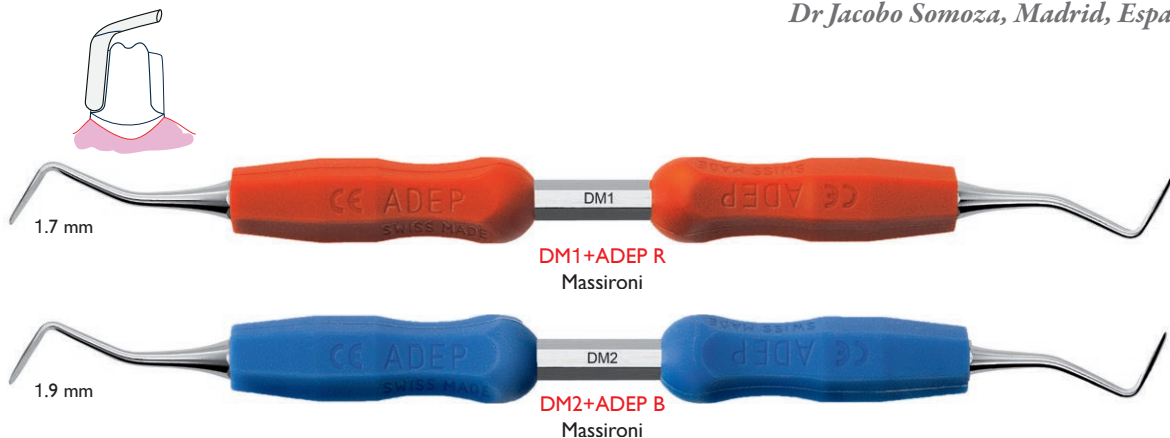
“The perfect precision instruments for sharp & clear preparation margins”

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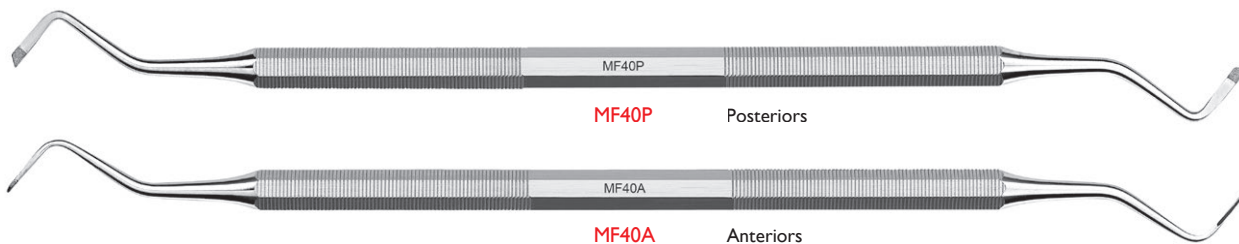
Thanks to the diamond-coated hand  
instruments the margins are evenly  
levelled.

**"A unique & smart tool for those hard to reach  
inter-proximal margins"**

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High quality cement spatulas of hardened  
stainless steel. These instruments are  
known for their extremely long lifetime.  
Their perfectly flat blades allow a very  
efficient mixing.

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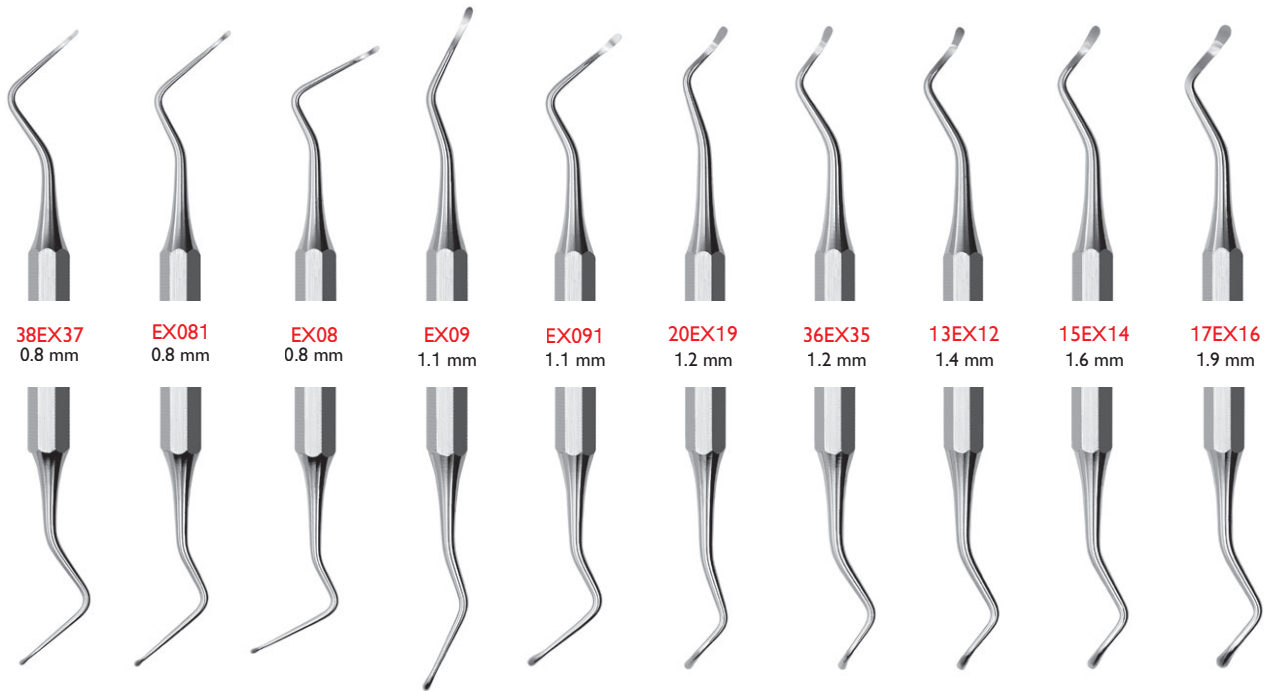
Our excavators have particularly sharp blades, free of burrs and perfectly rounded.

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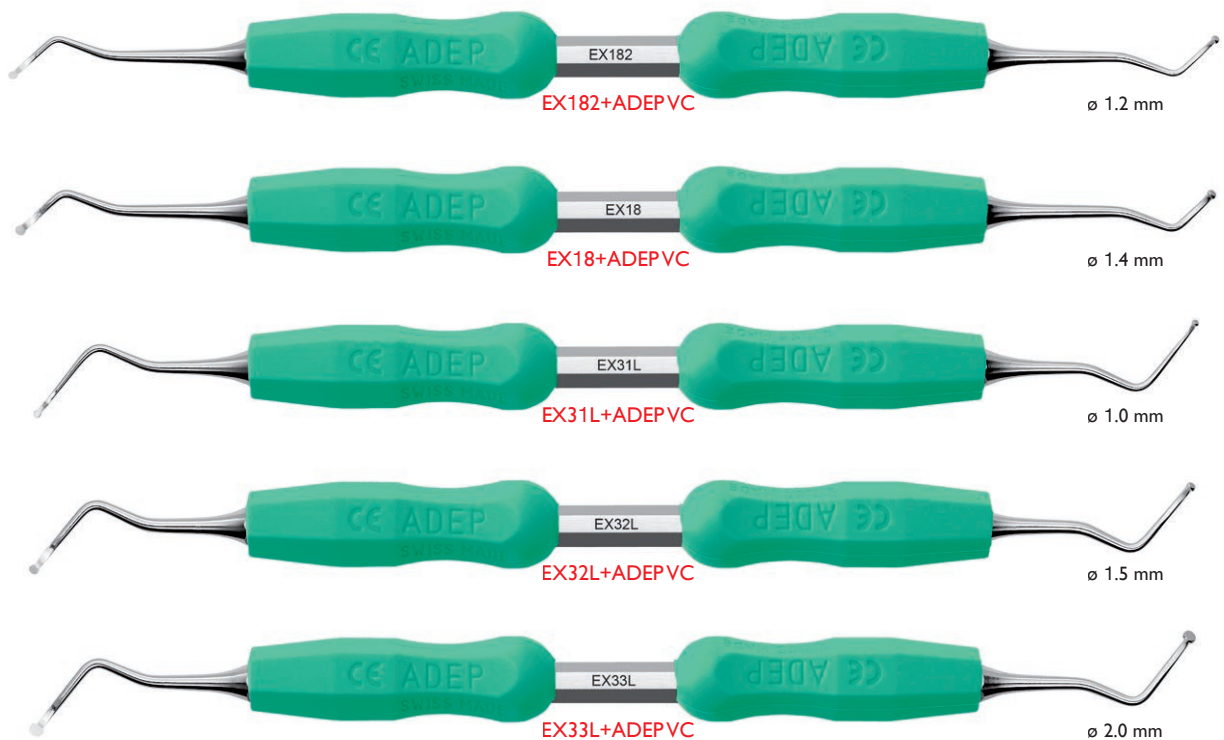
The quality of our steel and our manufacturing methods ensure a very low adhesion coefficient.

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This high tech coating offers a very low friction coefficient and high resistance to abrasion. In addition, its dark colour prevents the clinician from being hindered by the reflection of the steel. The Shadeline® coated instruments allow an easier handling and a more fluid shaping of diverse composite materials.

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This coating has proven non-adherence properties.

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The instruments marked with CT X\* are available in the 3 finishings.

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Important: Clean the tip immediately after applying composite to remove possible film. Do not clean with a hard bristle brush. Fine scratches make it easy for composite to adhere to the surface.

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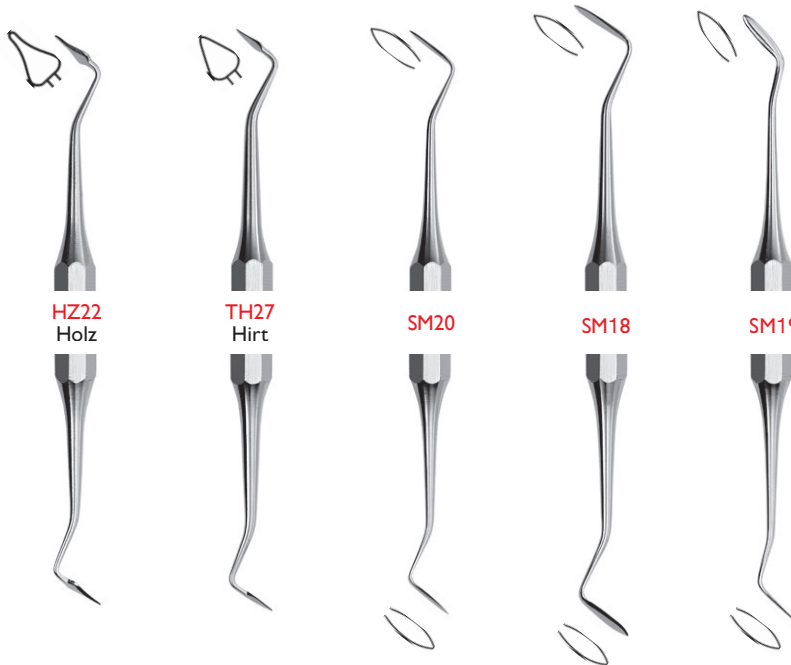
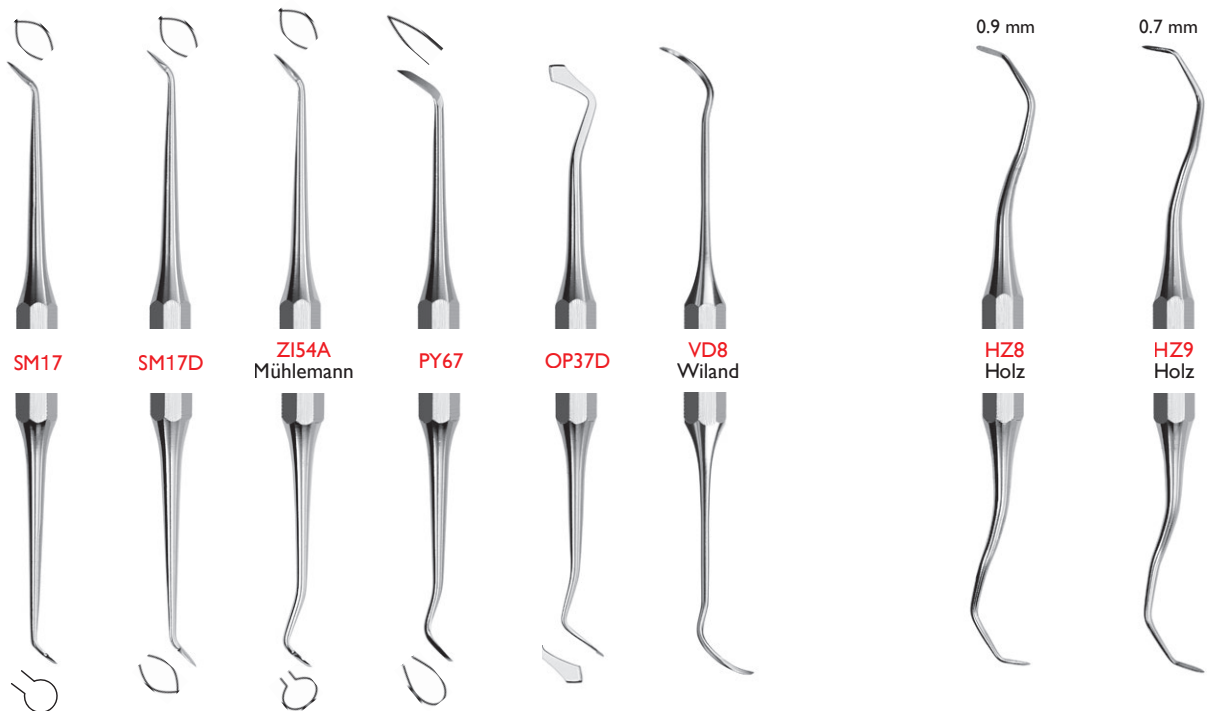
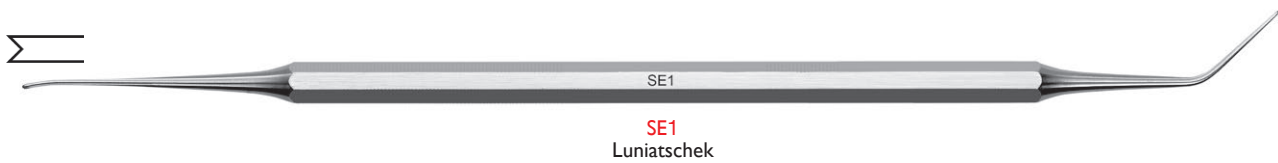
Do not damage your scalers, use the proper instrument.

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Extremely fine explorers, diameter 0.15 mm and 0.25 mm for the detection of fissures in the scellment of the composite.

MATRIX

> 0.025 mm  
> 0.050 mm  
> 0.100 mm

150/250EX KIT

> 2x 150EX & 250EX + 3x MATRIX

Hickel et al., FDI World Dental Federation: clinical criteria for the evaluation of direct and indirect restorations—update and clinical examples, Clin Oral Invest (2010) 14:349–366

CLEANext Line	CLEANext Line	CLEANext Line	CLEANext Line	58
> CLEANext B15	> CLEANext B15	> CLEANext B15	> CLEANext B15	58
> CLEANext B10	> CLEANext B10	> CLEANext B10	> CLEANext B10	58
> CLEANext B5	> CLEANext B5	> CLEANext B5	> CLEANext B5	58
> CLEANext kits	> CLEANext Kits	> CLEANext kits	> CLEANext kits	59

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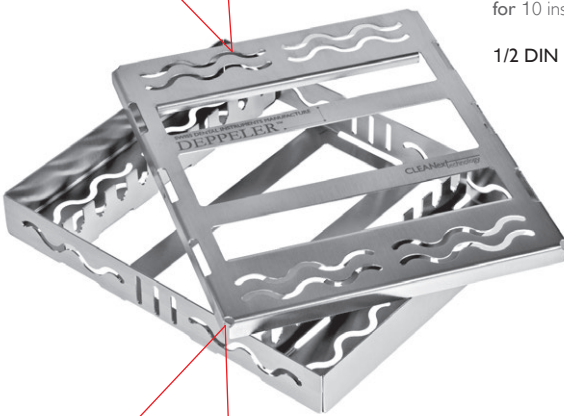
Our sterilization cassettes are the only ones to ensure 3D hygiene. They offer the following features:

1. total permeability
2. guaranteed retention of the instruments
3. closing system without hinges.

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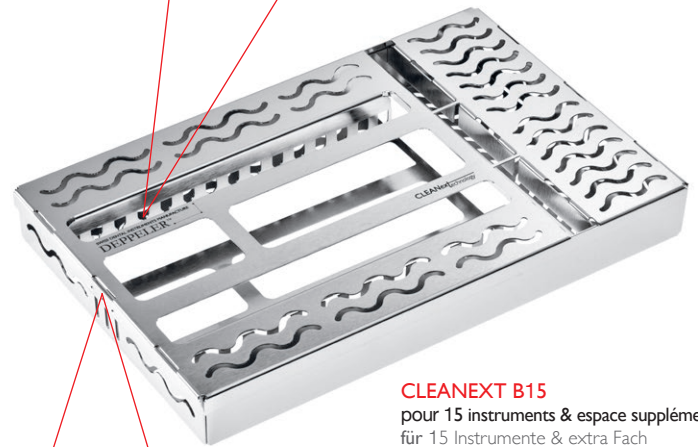
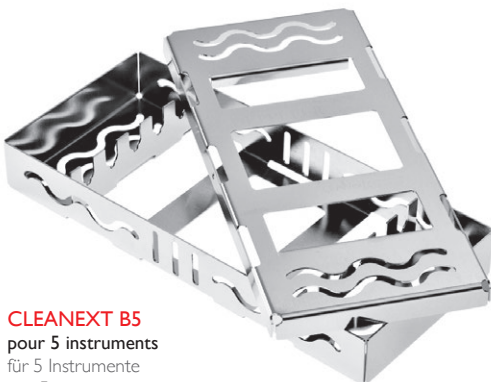
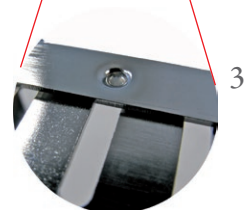
#### Handles

The kits are available with different handles, see p. 2. Please adapt the product references.

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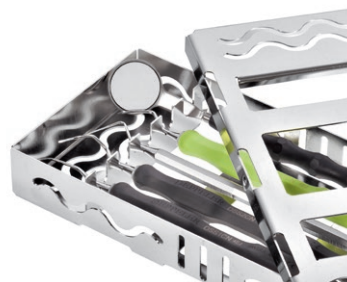
**Handles**

These kits are available in classic or with ADEP.

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### For the titanium curettes

#### Principle

Titanium is a soft material, it is essential to sharpen the curettes to keep optimum efficiency. Wear of cutting edge should in no way be offset by more force; your curettes could bend.

#### Maintenance

The sharpening of titanium curettes should be done on a stone specifically designed for this material (our reference : ESS4). This sharpening stone is ideally suited to avoid burr formation during sharpening. On the other hand, never use a stone having re-sharpened steel curettes. Steel particles which are placed there could damage the titanium. Desinfection & sterilisation of these instruments must be carried out in accordance with current standards.

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### For the ADEP silicone handles

#### Fitting

Dip the silicone sleeves into Petroleum Spirit (Benzinum Medicinale) for about twenty seconds, one by one. Then slide them on to the instrument handle, one by one. Protect the tip of the blade with a piece of electric wire insulating to avoid scratching the inside of the sleeve. This process ensures a hermetic fit. Do not immerse the sleeves in the Petroleum Spirit for more than 45 seconds. Petroleum Spirit vapors are highly inflammable. Keep away from sparks or an open flame. The sleeves are not meant to be re-used on further instruments.

#### Maintenance

When disinfecting and washing, respect the producers recommended concentrations for the products used. Do not exceed the recommended times. Rinse very carefully afterwards to prevent discoloration of the handle sleeves. They comply with the ISO standard 17665 for the sterilization of medical devices.

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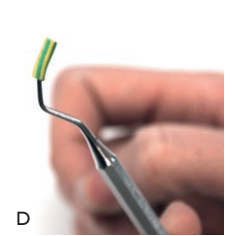
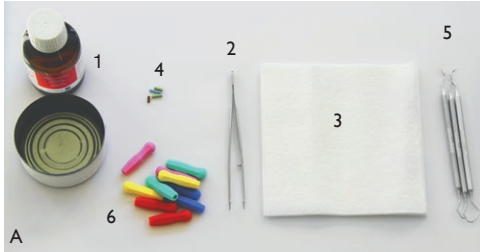
B. Dip the ADEP sleeve into the Petroleum ether for 20 to 45 seconds. Attention: for an optimal result, do not surpass 45 seconds.

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C. Take the ADEP sleeve with the tweezers, drain it a little and wipe it dry on the cloth.

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D. Slip a piece of wire insulating over the tip of the instrument to protect the ADEP sleeve.

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E. Hold the ADEP sleeve like on the picture and slide it along the instruments curve. Press a little with your ring finger on the back of the instrument in order to better follow the curve. Repeat the procedure with the second sleeve.

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Pay attention to place the sleeves correctly. Control it with the bellow pictures.

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By following exactly these instructions, your ADEP sleeves will fit perfectly for the whole instrument's life. The sleeves are not meant to be re-used on further instruments.

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